First Aid Plasters Mix en

Rich mix of plasters in a variety of shapes and sizes to help you stay prepared.

Warnings:

This is a single use device. Reuse may result in compromising product integrity or lead to device failure.

First Aid Plasters Mix includes the following CE marked Medical Devices:

Ultra Stretch Flexible Comfort Bandages

Flexible, 4-way stretch, to cover and protect minor wounds

6 22 mm x 57 mm
4 28 mm x 76 mm
1 10 cm x 6 cm
2 44,5 mm x 51 mm

Made in Poland MD C E UK CA

3M Deutschland GmbH Health Care Business Carl-Schurz-Str. 1, 41453 Neuss, Germany

Aqua Clear Waterproof Bandages

To cover and protect minor wounds

3 26 mm x 57 mm
2 60 mm x 88 mm

Made in USA with globally sourced materials MD C E UK CA

3M Company 2510 Conway Ave, St. Paul, MN 55144 USA EC REP 3M Deutschland GmbH Health Care Business Carl-Schurz-Str. 1 41453 Neuss, Germany

Steri-Strip™ First Aid Skin Closures

Steri-Strip™ Skin Closures are adhesive strips used to secure, close and support small cuts and wounds. To be used as directed by physician or health care provider for treatment of small cuts and wounds in conjunction with skin sutures and staples or, after their removal, for wound support. Holds tight as your cut heals, breathable.

Directions for use:

- Clean and dry skin. Remove excess hair.
- Fold one end of the liner and tear off the strips separately.
- Draw together the skin edges with fingers. Apply strip without tension using gentle, but firm pressure. Squeeze from the center outward.
- Space additional strips approximately 3mm apart.
- Additional strips can be applied parallel to the wound and approximately 12 mm from the closure ends to improve adhesion.
- To remove: gently peel skin closure low and slow, towards the center of the cut, while supporting the skin.

Contraindications:

- Do not use where good adhesion cannot be obtained due to the presence of wound fluid, skin oils, moisture, or hair.
- Do not use on puncture, dirty, or infected wounds. Consult your health care provider if you notice any of these signs of infection: fever, pain, swelling, itching, rash or burning.
- Do not use in high tension wounds which cannot be easily held together with fingers.
- Do not use as a security device, such as for holding tubing in place.

Warnings: To reduce the risk of infection, do not reuse strips.

Precaution:

- As with all adhesive products applied to the skin, a small percentage of individuals may experience skin irritation, discoloration, or injury during wear or upon removal.
- Tension caused by post-operative swelling, body/joint movement or applying skin closures under tension may cause skin injury, including blistering or loss of adhesion.
- Use with additional adhesive products may cause blisters or skin injury.
- If the wound is deep and bleeding continuously, seek medical assistance immediately.
- This product is not designed, sold or intended for use except as indicated. Please report a serious incident occurring in relation to this device to 3M and the local competent authority.

Always read the label. Use as directed. If discomfort or pain results, persists or increases, discontinue use and consult a physician.

Storage/Shelf Life/Disposal

Store at room temperature. Avoid heat and humidity. Do not use if expired or if expiration date is missing or illegible. Dispose of per local guidelines.

Date of issue 2022-08, rev. 3

2 3 x 6 mm x 75 mm REF H1541 3M Necare™ Steri Strip™ First Aid Skin Closures
MD C E 2797
Made in USA with globally sourced materials
3M Company 2510 Conway Ave, St. Paul, MN 55144 USA EC REP 3M Deutschland GmbH Health Care Business Carl-Schurz-Str. 1 41453 Neuss, Germany

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Pansements Ultra Stretch Flexible Comfort

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