Nexcare

Steri-Strip™ First Aid Skin Closures


en

Steri-Strip® Skin Closures are adhesive strips that are secure, close and support small skin cuts and wounds. To be used as directed by physician or health care provider for treatment of small skin wounds in conjunction with skin sanitisers and disinfectants or after their removal, for wound support. Do not use on deep or jagged cuts, on hot or wet skin, on facial, breakable.

Directions for use:

- 1 Clean and dry skin. Remove excess hair.
- 2 Fold one end of the liner and strip off the sticky separator.
- 3 Smooth together the skin edges with fingers. Apply strip without tension using gentle, firm pressure. Smooth from the center outward.
- 4 Smooth strip edges approximately 3mm apart.

Additional closures may be applied parallel to the wound and overlapping it 12 mm from the closure ends to improve adhesion. To remove gently peel skin slowly and slow, towards the center of the skin, while supporting the skin.

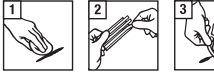Contraindications:

- Do not use where good adhesion cannot be obtained due to the presence of wound fluid, skin oils, moisture, or hair.
- Do not use on puncture, sting, or infected wounds. Consult your health care provider if you notice any of these signs of infection: fever, pain, swelling, itching, or burning.
- Do not use on high tension wounds which cannot be easily held together with fingers.
- Do not use on a securement device, such as on holding band in place.

Warnings: To reduce the risk of infection, do not use tearsips.

Precaution:

- As with all adhesive products applied to the skin, a small percentage of individuals may experience skin irritation, discoloration, or injury during wear or upon removal. * Remove caution by post-application, body joint movement or applying skin closure, under tension may irritate skin, including blistering or loss of adhesion. * Use with additional adhesive products, may cause blisters or skin injury. * If the wound is deep and bleeding continuously seek medical assistance immediately. * This product and its removal is not intended for use except in first aid.

Please report a serious incident occurring in relation to this device to 3M and the local competent authority.

Always read the label. Do not use. If discomfort or pain results, persist or increase, discontinue use and consult a physician.

© 2012 3M Corp. 2012-06

Storage/ Shelf life/ Disposal

Store at room temperature. Avoid heat and humidity. Do not use if expired or if expiration date is missing or illegible. Dispose of after local guidelines.

Made in USA with globally sourced materials

3M Company
2510 Cornway Ave
St. Paul, MN 55144 USA

3M Deutschland GmbH
Health Care Business
Carl-Schurz-Str. 11
41453 Nettesheim, Germany

For more information, see:
HCS@Germany.3m.com
3M and Nexcare are trademarks of 3M.
© 2022, 3M. All rights reserved.
Date of issue 2022-06

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