

Adper™ Single Bond Plus Adhesive

- Hold the shaft of the disposable applicator over the opening where the applicator enters the foil package with your thumb and index finger.

Do not activate the unit dose without a disposable applicator.



- With your thumb and index finger of the other hand, squeeze the large blister to transfer the adhesive into the chamber enclosing the applicator.



- Briefly spin the applicator to fully saturate with adhesive.



Optional

- Prior to dispensing adhesive, the applicator may be bent in the foil package to enhance intra-oral access.
- Carefully depressing the unit dose as shown facilitates rewetting applicator with adhesive.



Adper™ Single Bond Plus Adhesive



- Pinch the sides of the cap to release the locking mechanism.



- Flip the cap back to reveal the dispensing tip.



- Squeeze out the exact number of drops desired into the mixing well.



- When finished, flip the cap back until it is secured by its locking mechanism.