

3M™ Wound Cleanser

Product Use Information

Product Description:

3M Wound Cleanser is a non-antimicrobial, no-rinse liquid with a zinc-nutrient formulation that provides a moist, mildly acidic environment conducive to wound healing. It has a low toxicity potential, is pH balanced and fragrance free. 3M Wound Cleanser is delivered in an easy open, easy squeeze bottle.

Intended Use:

3M Wound Cleanser is designed to cleanse partial and full thickness ulcers and dermal wounds including:

- pressure ulcers
- abrasions
- superficial partial thickness burns
- open surgical incisions
- venous ulcers
- arterial ulcers
- skin tears
- neuropathic ulcers

Directions for use:

- 1) Twist cap open.
- 2) Hold bottle approximately one inch from wound bed. Aim nozzle at wound and squeeze bottle, directing stream of cleanser along the base and sides of the wound. Rinsing is not needed.
- 3) Blot up excess moisture with a gauze pad.
- 4) Dry surrounding skin.

Follow facility/institution guidelines for infection control

Caution:

For external use only

Ingredients:

Water, dextrose, sodium chloride, preservatives and nutrients associated with wound repair (pyridoxine hcl, zinc acetate, zinc chloride, zinc sulfate).



Health Care
3M Center, Building 275-4W-02
St. Paul, MN 55144-1000
USA
1 800 228-3957
www.3M.com/healthcare

3M Canada Inc.
Post Office Box 5757
London, Ontario N6A 4T 1
Canada
1 800 563-2921

©3M 2004
September 2004
70-2009-6392-7 (POD)