

# 3M™ Incise Drapes


## 3M™ Ioban™ 2 Antimicrobial Incise Drape – 6661EZ

**The drape that fits. Accept nothing less.**

3M Ioban 2 Antimicrobial Incise drapes have been preferred by surgeons for over 25 years to help manage the risk of surgical site infections.

Now we've taken the unique "picture frame" delivery system from our popular 3M™ Tegaderm™ Transparent Dressings and incorporated it into the new 3M™ Ioban™ 2 Antimicrobial Incise Drape – 6661EZ, for minimally invasive surgical procedures using implantable devices and/or materials.

The 6661EZ Ioban 2 Drape is identical in clinical performance and efficacy to our other Ioban 2 Antimicrobial Incise drapes, and offers the additional advantage of quick, easy and precise drape placement every time. Its size and design also allow for time-saving and convenient one-person drape application.

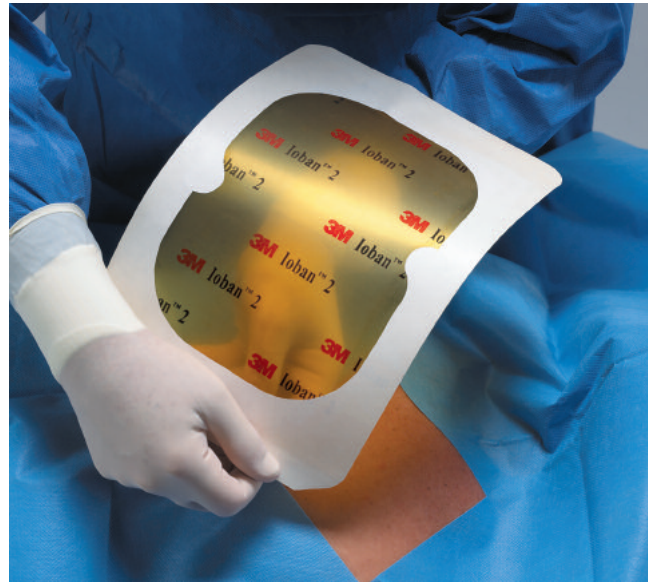


- Unique framed delivery system enables simplified, one-person application (see below)
- Size (10.5" x 8") is ideally suited for high-volume minimally invasive implant procedures
- Iodophor-impregnated adhesive provides continuous, broad-spectrum antimicrobial activity, and superior adhesion to the skin and wound edge
- Strong, conformable and breathable polyester incise film provides a sterile surface, and helps to decrease bacterial contamination from skin to gloves, gowns, instruments and implants




① Grasp tabs and align drape over incision site.



② Adhere drape to the sterile field as release liner is peeled away.

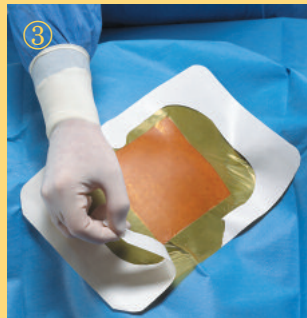

③ Remove frame.

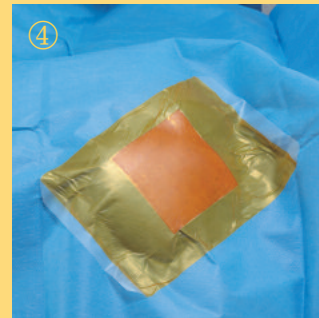

④ Begin procedure.

3M Medical


from the 3M Health Care family



# 3M™ Ioban™ 2 Antimicrobial Incise Drape – 6661EZ

## Ordering Information for 3M™ Ioban™ 2 Antimicrobial Incise Drape – 6661EZ

Catalog Number Product Description Size	Packaging	UPN	Minimum Order	Pricing Unit
<b>6661EZ</b> 3M™ Ioban™ 2 Antimicrobial Incise Drape – 6661EZ; sterile individually packaged Dimensions: 10.5" x 8" (26 cm x 20 cm)	50 Drapes/Box 2 Boxes/Case	30707387567164 50707387567168	1 Case	Case
<b>6661EZSB</b> (sterile bulk) <i>Custom Packs Only</i> 3M™ Ioban™ 2 Antimicrobial Incise Drape – 6661EZ; sterile bulk packaged; sterile Dimensions: 10.5" x 8" (26 cm x 20 cm)	300 Drapes/Case	50707387567175	1 Case	Case

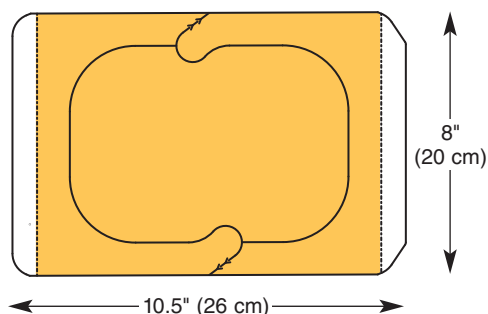

### For more information

For more information about 3M surgical drapes or other 3M Health Care products:

- Contact your local 3M representative
- Call the 3M Health Care Customer Helpline: **1-800-228-3957**
- Visit the 3M Health Care products Web site: [www.3M.com/infectionprevention](http://www.3M.com/infectionprevention)

**3M Infection Prevention Solutions. A vital step in infection prevention.**



### Health Care

3M Center, Building 275-4W-02  
St. Paul, MN 55144-1000  
U.S.A.  
1 800 228-3957

[www.3M.com/healthcare](http://www.3M.com/healthcare)

### 3M Canada Company

Post Office Box 5757  
London, Ontario N6A4T1  
Canada  
1 800-563-2921

Outside of USA, please contact your 3M Representative



40% Pre-consumer waste paper  
10% Post-consumer waste paper

Printed in U.S.A.  
© 3M 2006 All Rights Reserved  
70-2009-7131-8