

3M™ Cavityon™ No Sting Barrier Film

Incontinence Skin Care Instructions



3M™ CAVILON™ NO STING BARRIER FILM

Incontinence Skin Care Instructions



Step 1

Gather supplies:

- A pH-balanced skin cleanser, such as 3M™ Cavilon™ No-Rinse Skin Cleanser
- 3M™ Cavilon™ No Sting Barrier Film
- Absorbent product (e.g., brief or pad)
- Soft cloth(s)
- Gloves

Step 2

Remove soiled brief or pad.



Step 3

Cleanse and dry skin.

Moisten cloth with skin cleanser.



Spray skin with cleanser, then wipe skin with moistened cloth to remove soil. Repeat as needed until skin is clean. Rinsing is not required for Cavilon No-Rinse Skin Cleanser. If using other skin cleansers, for best results, rinse completely with water and dry thoroughly.

Make sure skin is clean and dry before going to Step 4.



Step 4

Apply Cavilon No Sting Barrier Film beginning at anal area.

Apply an even coating of film while moving outward over buttocks and other areas of exposure such as external genitalia or groin folds. Let film dry completely (approximately 30 seconds). If an area is missed, reapply to that area only after the first application of film has dried. Hold skin folds apart until completely dry.

Reapply every 24 to 72 hours. Apply more often if:

- Skin becomes red or
- Incontinence worsens (e.g., increased frequency and/or liquid stools)

Avoid applying the film more than every 12 hours.

Step 5

Apply a clean absorbent product.

Step 6

Document Care:

Report redness that does not improve. Consider influence of friction or pressure if area is over a bony prominence.

Document care per facility protocol.

Ordering Information

Catalog No.	Product	NDC/NHRC No.	Size	Items/Box	Boxes/Case	HCPCS Code
3343	3M™ Cavilon™ No Sting Barrier Film	8333-3343-01	1.0mL wand	25	4	A6250
3344	3M™ Cavilon™ No Sting Barrier Film	8333-3344-01	1.0mL wipe	25	4	A6250
3345	3M™ Cavilon™ No Sting Barrier Film	8333-3345-01	3.0mL wand	25	4	A6250
3346	3M™ Cavilon™ No Sting Barrier Film	8333-3346-01	28.0mL spray	12	1	A6250

Disclaimer:

HCPCS codes have been provided to assist you in the preparation of insurance claims. Please note, however, that the reimbursement information provided by 3M Health Care and its representatives is intended to provide general information relevant to coverage and coding for 3M products. Insurers' reimbursement policies can vary and the use of the codes discussed here does not guarantee that an insurer will cover or pay at any particular level. Health care providers should exercise independent clinical judgment in choosing the codes which most accurately describe the products provided.

Convenient. Comforting. Trusted.

To learn how the full line of Cavilon Professional Skin Care products can help simplify and improve your skin care program, visit www.3M.com/Cavilon, contact your 3M Skin & Wound Care Representative or call the 3M Helpline at 1-800-228-3957. Outside of the United States, contact the local 3M subsidiary.



**Skin & Wound Care Division
3M Health Care**

2510 Conway Avenue
St. Paul, MN 55144
USA
1-800-228-3957
www.3M.com/Cavilon

3M and Cavilon are trademarks of 3M.
Please recycle. Printed in U.S.A.
© 3M 2012. All rights reserved.
70-2009-4266-5