



January 15, 2007

**USER ADVISORY NOTICE**  
**3M™ Bitrex Fit Test & Sensitivity Solutions**

Dear Valued 3M Customer:

We recently identified that certain fit testing and sensitivity solutions used in the 3M™ FT-30 Qualitative Fit Test Apparatus may not have contained suitable levels of the active ingredient denatonium benzoate, or Bitrex™. Solutions potentially affected were manufactured in 2004 and are identified by the lot numbers 41008 and 41009 only.

If you have one or both of these lot numbers, we recommend you stop using for fit testing, and re-fit test using solutions manufactured in 2005 or after. The year of manufacture may be determined from the first digit of the lot number (e.g., lot 5xxxx manufactured in 2005).

3M remains committed to providing only the highest quality products for our customers. Therefore, if you have bottles with these lot numbers we request that you dispose of them, and exchange as outlined below:

The following 3M Bitrex Fit Testing solutions are affected:

<u>ID</u>	<u>Year</u>	<u>Lot Number</u>
FT31 Sensitivity Solution	2004	41008
FT32 Fit Test Solution	2004	41009

The lot number is stamped on the bottom of the bottle.

Please fill out the attached form and fax to 1.800.503.4581. A 3M representative will promptly send you replacement Bitrex Solutions. This information is also available on our Website at [www.3m.com/Occsafety](http://www.3m.com/Occsafety).

3M regrets any inconvenience this may cause. If you have any questions regarding this issue, please call 3M Technical Service at 1.800.243.4630 or for Return Questions call 1.800.831.0658 or log onto our Website at [www.3m.com/Occsafety](http://www.3m.com/Occsafety).

Page 2

Sincerely,

A handwritten signature in black ink that reads "Mike Martinez". The signature is written in a cursive, slightly slanted style.

Michael Martinez  
Sales Director

A handwritten signature in black ink that reads "Paula Lien". The signature is written in a cursive, slightly slanted style.

Paula Lien  
Quality Assurance Director

Attach.

**3M Bitrex USER FAX BACK FORM**

***FAX TO:***

**To: 3M Bitrex Exchange Coordinator**

**Fax: 1.800.503.4581**

Instructions:

1. Completely fill out this form and fax it to the number listed above
2. Upon receipt of FAX, 3M will immediately ship replacements

Company Name\*: \_\_\_\_\_

Contact Name\*: \_\_\_\_\_

Address\*: \_\_\_\_\_ (no P.O. boxes please)

City\*: \_\_\_\_\_ State\*: \_\_\_\_\_ Zip Code\*: \_\_\_\_\_

Telephone: \_\_\_\_\_

Fax\* : \_\_\_\_\_ e-mail\*: \_\_\_\_\_

<u>ID</u>	<u>Year</u>	<u>Lot Number</u>	<u>Number of Bottles</u>
FT31 Sensitivity Solution	2004	41008	
FT32 Fit Test Solution	2004	41009	

**Free replacement applies only to FAX forms returned before March 30, 2007**

*Please call 1.800.831.0658 with any questions regarding this exchange process*

\* : Mandatory Information