

3M™ Wound Cleanser

Easy to use, gentle wound cleanser with zinc-nutrient formulation.

Gentle

New 4 oz. Size



Safe Cleansing

3M Wound Cleanser is a non-antimicrobial, no-rinse liquid with a zinc-nutrient formulation that provides a moist, mildly acidic environment conducive to wound healing.

- Non-cytotoxic, pH balanced
- Gentle spray stream consistent with guidelines for safe and effective cleansing
- Delivered in a convenient easy open, easy squeeze bottle
- Fragrance free, no rinse formulation



3M™ Wound Cleanser 91101

3M Wound Cleanser is available in a single patient multidose unit, designed to cleanse partial and full thickness ulcers and dermal wounds including:

- pressure ulcers
- abrasions
- superficial partial thickness burns
- open surgical incisions
- venous ulcers
- arterial ulcers
- skin tears
- neuropathic ulcers

Directions for use:

1. Twist cap open.
2. Hold bottle approximately one inch from wound bed. Aim nozzle at wound and squeeze bottle, directing stream of cleanser along the base and sides of the wound. Rinsing is not needed.
3. Blot up excess moisture with a gauze pad.
4. Dry surrounding skin.



Follow facility/institution guidelines for infection control.

Caution: For external use only

Ordering Information

3M™ Wound Cleanser

Catalog Number	HCPCS Code*	Size	Bottles Per Case
91101	A6260	4 oz. squeeze bottle	12

*HCPCS codes as referenced in the PSC Local Coverage Determination Surgical Dressings and Ostomy Policies.

Disclaimer: HCPCS codes have been provided to assist you in the preparation of Medicare Part B claims. Please note, however, that the reimbursement information provided by 3M Health Care and its representatives is intended to provide general information relevant to coverage and coding for 3M products. Insurers' reimbursement policies can vary and the use of the codes discussed here does not guarantee that an insurer will cover or pay at any particular level.



Medical Division
3M Health Care
3M Center, Building 275-4W-02
St. Paul, MN 55144-1000
U.S.A.
1-800-228-3957
www.3M.com/healthcare

Medical Division
3M Canada
P.O. Box 5757
London, Ontario N6A 4T1
Canada
1-800-364-3577

3M is a trademark of 3M.
Please recycle. Printed in USA.
© 3M 2007,2010. All rights reserved.
70-2009-7536-8