

Description:

3M Wound Cleanser is a non-antimicrobial, no-rinse liquid with a zinc-nutrient formulation that provides a moist, mildly acidic environment conducive to wound healing. It has a low toxicity potential, is pH balanced and fragrance free. 3M Wound Cleanser is delivered in an easy open, easy squeeze bottle.

Indications for Use:

The product is designed to cleanse partial and full thickness ulcers and dermal wounds including:

- pressure ulcers
- abrasions
- superficial partial thickness burns
- open surgical incisions
- venous ulcers
- arterial ulcers
- skin tears
- neuropathic ulcers

Ingredients: Water, dextrose, ethyl alcohol, sodium chloride, methylparaben, pyridoxine hydrochloride, propylparaben, zinc acetate, zinc chloride, zinc sulfate

Contraindications: None known

Warnings: None known

Precautions

1. Treatment of any wound should be part of a well-defined plan and under the supervision of a health care professional.
2. Observe the wound for signs of infection. Consult a health care professional if any of the following are noted: fever, increased pain, redness, bleeding, swelling, unusual odor, increased drainage or cloudy or foul drainage.
3. If the wound does not begin to show signs of healing or if any other unexpected symptoms occur, consult a health care professional.

Directions for Use

1. Twist cap open.
 2. Hold bottle approximately one inch from wound bed. Aim nozzle at wound and squeeze bottle, directing stream of cleanser along the base and sides of the wound. Rinsing is not needed.
 3. Remove debris and blot up excess moisture with a gauze pad.
 4. Dry surrounding skin.
- Follow facility guidelines for infection control.

Storage and Shelf Life

For best results, store at room temperature. Avoid excessive heat and humidity. For shelf life, refer to the expiration date, which is printed on each package.

How Supplied

Individually packaged for single patient use only.

If you have any questions or comments, in the USA contact the 3M Health Care Customer Helpline at 1-800-228-3957.

In Canada, contact 3M Canada, Inc., PO Box 5757, London, Ontario, N6A 4T1,

1-800-268-9696

Outside of the United States, contact your local 3M subsidiary.

Explanation of symbols



- Attention, see instructions for use.



- The lot in a box and the hourglass symbols represent lot number and expiration date. The hourglass is followed by a year and month which represents the expiration date (year and month: 2010-10). The entire line after the hourglass represents the lot number. (2010 - 10 AZ).

Distributed by

3M Health Care

St. Paul, MN 55144-1000

(U.S.A.) 1-800-228-3957

www.3M.com

38-9018-4699-8