

3M™ Steri-Drape™ and Steri-Drape™ 2 Incise Drapes

WARNING: Do not defibrillate through plastic drapes as arcing or ineffective defibrillation may result.

Precaution: Do not resterilize.

Indications: 3M Steri-Drape and Steri-Drape 2 incise drapes are indicated for use as incise drapes to cover patients' skin. They are intended for external use only.

Directions for Use:

1. Disinfect the surgical or operative site according to hospital policy or physician-prescribed routine.
2. Make sure skin is dry.
3. Apply Steri-Drape or Steri-Drape 2 incise drape to thoroughly dry intact skin. Hold incise drape a few inches above the skin. Grasp edges of non-adhesive film handle. Firmly peel off paper liner (see figure a).
4. When "stop" symbol appears, discontinue peeling (see figure b).
5. Using a sterile towel, press down firmly on the film, first contacting the skin along the intended incision line to ensure good skin contact (see figure c). Firmly smooth film into place **WITHOUT TENSION**, working away from the intended incision line to achieve wrinkle free adhesion. Peel remainder of drape off paper liner and complete drape application.
6. To remove, gently peel back drape at 180 degree angle from the skin. This means folding the film back on itself and maintaining that angle while pulling the film away from the skin (see figure d & e). During the removal process hold onto the film with the thumb and hand as close as possible to the junction of skin and adhesive film. This prevents severe stretching. Skin of elderly patients is typically fragile and thus requires even greater care when applying and removing adhesive products such as drapes.

Sterile unless package is damaged or open.

Storage Conditions: For best results, store at room temperature. Avoid excessive heat and humidity.

Explanation of Symbols



- Attention, see instructions for use.



2010-10 AZ

- The lot in a box and the hourglass symbols are symbols that represent lot number and expiration date. The hourglass is followed by a year and month which represent the expiration date (year and month: 2010-10). The entire line after the hourglass represents the lot number (2010-10 AZ).



- Sterile, method of sterilization by radiation



- Do not reuse



- This product and package do not contain natural rubber latex.

CE 0086

3M Health Care

D-41453 Neuss, Germany

3M

Made in USA by

3M Health Care

St. Paul, MN. 55144-1000
(U.S.A.) 1-800-228-3957

Made in Mexico by


3M Health Care

St. Paul, MN. 55144-1000
(U.S.A.) 1-800-228-3957


Cat Nos. 1037, 1040, 1050, 1051,
2037, 2040, 2045, 2050, 2051

Cat Nos. 1035, 2035


a.

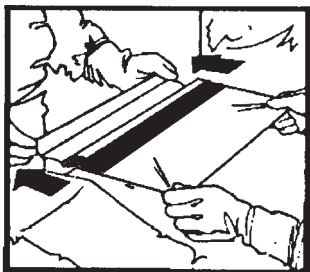


b.

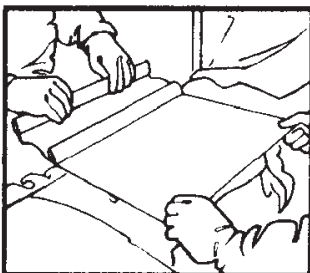


c.

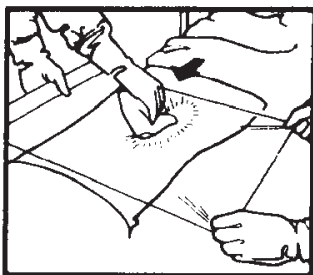

d.

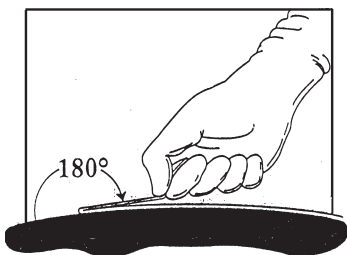

e.

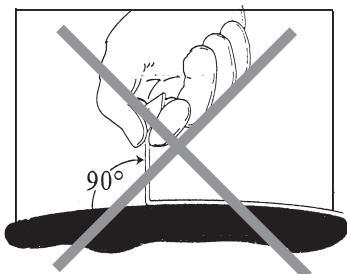

Artwork #: 38-9001-5910-4

Date: 10/03/03

Author: R Barker

Supersedes: 38-9001-3883-5

Pkg. Spec. Reference: 34-7053-1700-5

Structure: Ins

SCALE:  **1 Inch**

all artwork prints black