

3M Skin & Wound Care Division
3M™ Tegaderm™ IV Securement Dressing
Designed for the BD Nexiva™ Closed IV Catheter System

Tegaderm™ IV Transparent Film Dressings



**Secure.
Protect.
Stabilize.**

Tegaderm™ IV Dressing for the BD Nexiva™ Catheter System combines the simplicity of Tegaderm™ dressings with a new style of catheter to provide another dependable dressing you can trust.

- Designed specifically to reduce catheter dislodgement
- Judged by clinicians as an excellent alternative in decreasing catheter movement, dislodgements and fall-outs compared to current stabilization techniques*
- Better at conforming around the BD Nexiva™ Catheter System as compared to standard IV dressings
- Perceived by clinicians to wear longer than current standard film dressings*
- Transparent, breathable and impermeable to liquids, viruses** and bacteria

*3M internal data on file

***In vitro* testing has proven that Tegaderm™ and Tegaderm™ HP provide a viral barrier from viruses 27 nm in diameter, (e.g. HCV) or larger (e.g. HBV and HIV) while the dressing remains intact without leakage.



3M™ Tegaderm™ IV Securement Dressing

Designed for the BD Nexiva™ Closed IV Catheter System

Application Instructions for #9525HP



Step 1

Prep the catheter site according to your facility's protocol. Let all prep solutions dry completely. Remove liner. Place dressing, adhesive side up, on sterile side of dressing package.



Step 2

After catheter insertion, stabilize catheter with one hand. With other hand, grasp paper end of dressing, turn it adhesive side down and bend paper end slightly with thumb.



Step 3

Place dressing so that top edge of window lines up with bottom edge of catheter wings.



Step 4

Press down on slit side of dressing. Separate dressing slit at notched corner revealing a J-shaped slit.



Step 5

Pull extension tubing through the slit ensuring that tubing rests at end of J-curve. Overlap edges to close slit and press in place.



Step 6

Apply pressure over entire dressing surface.



Step 7

Slowly remove the frame while smoothing down dressing edge.



Step 8

Smooth the dressing from the center toward the edges, using firm pressure.



Step 9

Remove soft cloth tape from liner and apply to secure extension tubing.



Step 10

Record information on tape label and place it on or near dressing.

Removal Instructions for #9525HP



Step 1

Remove tape strips. Then gently grasp soft cloth on both sides of slit and separate.



Step 2

Slowly peel dressing back over itself while stabilizing catheter and supporting skin.

3M

Skin & Wound Care Division
3M Health Care
3M Center, Building 275-4W-02
St. Paul, MN 55144-1000
USA
1-800-228-3957
www.3M.com/healthcare

Medical Division
3M Canada
Post Office Box 5757
London, Ontario N6A 4T1
Canada
1-800-364-3577



BD, BD Logo and BD Nexiva are trademarks of Becton, Dickinson and Company.
© BD 2008.

3M and Tegaderm are trademarks of 3M. Please recycle. Printed in U.S.A.
© 3M 2009. All rights reserved.
70-2010-7264-5