Fitting Instructions

Must be followed each time respirator is worn.

- With nosepiece facing away from you, hold the bottom strap in each hand with the nosepiece up (Fig. 1).
- Position the respirator under your chin, pull the bottom strap over your head and position it around the neck, below the ears (Fig. 2).
- Pull the top strap over your head resting it high at the top of your head (Fig. 3).
- Place your fingertips from both hands at the top of the metal nosepiece. Using two hands, mold the nose area to the shape of your nose by pushing inward while moving your fingertips down both sides of the nosepiece (Fig. 4).
▲ Pinching the nosepiece using one hand may result in improper fit and less effective respirator performance. Use two hands.
- Perform a User Seal Check prior to each wearing. To check the respirator-to-face seal, place both hands completely over the respirator and exhale (Fig. 5). Be careful not to disturb the position of the respirator. If air leaks around nose, readjust the nosepiece as described in step 3. If air leaks at the respirator edges, work the straps back along the sides of your head. **If you CANNOT achieve a proper seal, DO NOT enter the contaminated area. See your supervisor.**

Removal Instructions

See step 2 of *Fitting Instructions* and cup respirator in hand to maintain position on face. Pull bottom strap over head. Still holding respirator in position, pull top strap over head and remove respirator.

Folding the Respirator: Avoid sharp bends on the metal nosepiece by placing your fingertips behind the nosepiece when folding the respirator.

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NIOSH Approved: N95

At least 95% filtration efficiency against solid and liquid aerosols that do not contain oil.

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3M
ST. PAUL, MN, USA
1-800-228-3957
9010, 9132, 9010CN, 9010MX Respirators

THESE RESPIRATORS ARE APPROVED ONLY IN THE FOLLOWING CONFIGURATIONS:

TC-	Protection ¹	Respirator				Cautions and Limitations ²
		9010	9132	9010CN	9010MX	
84A-4243	N95	X	X			ABCJMNOPS
84A-7770	N95			X		ABCJMNOP
84A-7879	N95				X	ABCJMNOP

1. PROTECTION

N95-Particulate Filter (95% filter efficiency level) effective against particulate aerosols free of oil; time use restrictions may apply

2. CAUTIONS AND LIMITATIONS

- A - Not for use in atmospheres containing less than 19.5 percent oxygen.
- B - Not for use in atmospheres immediately dangerous to life or health.
- C - Do not exceed maximum use concentrations established by regulatory standards.
- J - Failure to properly use and maintain this product could result in injury or death.
- M - All approved respirators shall be selected, fitted, used, and maintained in accordance with MSHA, OSHA, and other applicable regulations.
- N - Never substitute, modify, add, or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer .
- O - Refer to users instructions, and/or maintenance manuals for information on use and maintenance of these respirators.
- P - NIOSH does not evaluate respirators for use as surgical masks.
- S - Special or critical User's Instructions and/or specific use limitations apply. Refer to User's Instructions before donning.

3M
St. Paul, Minnesota, USA
1-800-243-4630
9010, 9132, 9010CN, 9010MX Respirateurs

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Rev D: 05-31-2019

This respirator contains no components made from natural rubber latex.

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<p style="text-align: center;">dZ</p> <p>Requester: Susan Anderson Creator: deZinnia_27085 File Name: 34872571517.indd Structure #: Artwork control Date: 04/07/20</p>	<p>Printed Colors – Front:</p> <p style="text-align: center;">● K</p>
	<p>Printed Colors – Back:</p> <p style="text-align: center;">● K</p>
	<p>Match Colors:</p>
<p>Scale:  1 Inch</p>	<p>This artwork has been created as requested by 3M. 3M is responsible for the artwork AS APPROVED and assumes full responsibility for its correctness.</p>