
3M ESPE

The TOTAL-ETCH ADHESIVE you trust now comes in our UNIT DOSE.

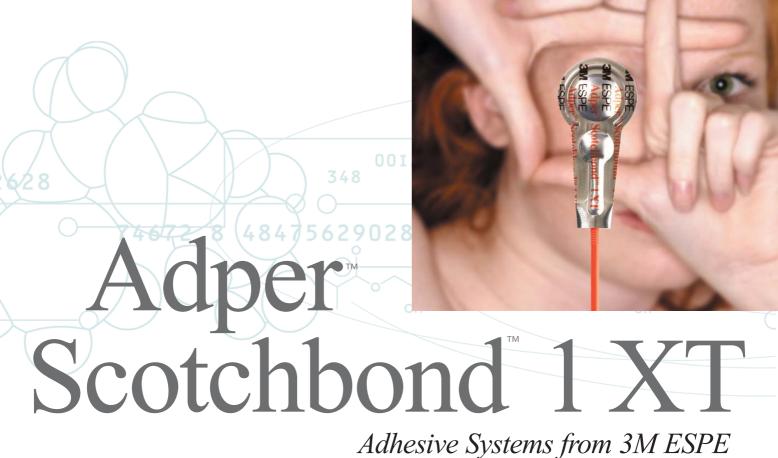

Performance backed by a proven adhesive.

Adper[™] Scotchbond[™] 1 XT Adhesive

Based on proven performance.

3M[™] ESPE[™] Adper[™] Scotchbond[™] 1 Adhesive is an ethanol/water based, 5th generation adhesive – widely known for excellent *in vitro* and *in vivo* clinical performance.

Three-Year Clinical Evaluation of Filtek P-60 Packable Composite Restorations

Objectives: The purpose of this prospective clinical trial was to assess the 3-year clinical performance of the packable composite Filtek P-60 (3M) in combination with the one-bottle adhesive Single Bond (3M) in Class II restorations. Methods: A total of 47 restorations were placed into Class II cavity prepations (27 premolars and 20 molars) of 33 patients (22 female/11 male; mean age 34) by one operator. Restorations were evaluated by two examiners at baseline, 1, 2 and 3 years according to the modified USPHS criteria observing the following characteristics: Retention Marginal Integrity, Anatomical Form,

Criteria influenced by Adper Scotchbond 1: Sur Retention, Marginal Integrity, Anatomical Form, Hea Surface Texture, Marginal Discoloration, exa Surface Staining, Post-op Sensitivity, Soft-SCO tissue Health, Maintenance of Interproximal reca Contacts and Secondary Caries. Chi-square only and Fisher's exacte tests were used for diff statistical analysis (P = 0.05). that

t-op Sensitivity, Soft-tissue
Caries. Chi-square and Fisher's
All the restorations received Alfa
ed post-op sensitivity. At 3 year
ncluded. From Baseline to 3 years
iety of reasons. No significant
Conclusion: The results indicate

of the restorations examined were clinically acceptable after 3 years of service.

Kiremitci A., "Three-year Clinical Evaluation of Filtek P-60 Packable Composite Restorations" J Dent. Res, 82 (Special Issue B), 195 (abstr. 1466) (2003).

Recently, 3M ESPE added nanofiller technology, similar to that first used in Filtek™ Supreme Universal Restorative. The resulting adhesive, Adper Scotchbond 1 XT, offers even higher bond

• Fast and easy – simply press one chamber and

strengths.

The unique unit dose

delivery system from

3M ESPE:

· Hygienic to use.

You have been asking for a TOTAL-ETCH adhesive in our UNIT DOSE ...and here it is!

• Reduce procedure time from start to finish. With the unit dose delivery, there is no measuring, mixing or cleanup.


• The unique design is self-contained, and holds everything from adhesive to brush-tip applicator.



The unique unit dose from 3M ESPE is easy to use.

Fast and convenient.

Using the unit dose is easy. Just squeeze the large blister to transfer the adhesive into the chamber enclosing the applicator. Briefly spin the applicator to fully saturate with adhesive and use. The technique is fast and convenient. When you're finished, just discard the applicator. There is no messy cleanup.

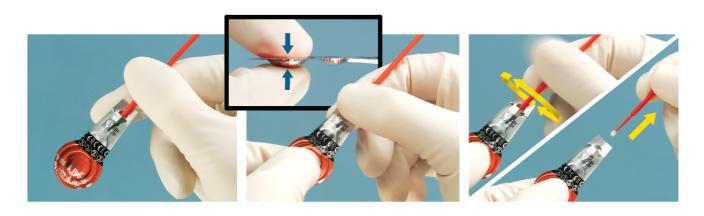


Now with nanofiller.

Assurance of consistency in every drop.

The extremely small particle size of nanofiller used in Adper Scotchbond 1 XT adhesive means the filler stays in suspension without the need of shaking.

Dentists not only save time by eliminating the shaking step, but they can have total confidence the adhesive is perfectly mixed – straight from the vial – every time.



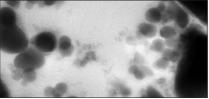

3M ESPE nanoparticles

competitor's filler

Using proprietary technology, the nanofiller in Adper Scotchbond 1 XT adhesive is incorporated into the adhesive as discrete, uniform-sized particles.

And, due to their extremely small size (just 5 nanometers in diameter), they stay in suspension – so, unlike other filled adhesives with larger-diameter particles, no shaking of the bottle is needed before use.




Source: 3M ESPE Laboratory test data



A stronger adhesive for improved performance.

Stronger than ever.

Adper Scotchbond 1 XT adhesive offers all the benefits of the clinically-proven Adper Scotchbond 1 adhesive, and now includes nanofiller technology. The improved adhesive provides higher bond strengths that can lead to more successful procedures and a whole new level of confidence for dentists and patients.

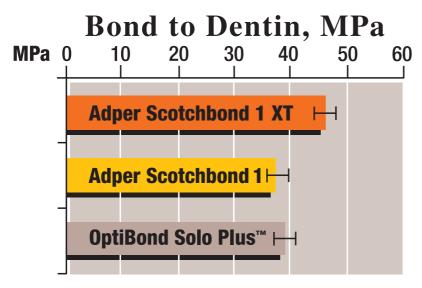



Microtensile bond evaluation shows significant improvement in dentin bond strength. DR. JORGE PERDIGAO (University of Minnesota, USA)

Excellent for all light-cured direct and indirect indications.

As with the original Adper Scotchbond 1 adhesive, Adper Scotchbond 1 XT total-etch adhesive is an advanced combination of glass ionomer and ethanol-based adhesive technology that shows excellent performance for:

- all classes of direct composite restorations, as well as root surface desensitization and porcelain veneers.*
- indirect restorations after light curing**— the film thickness measured less than 10 microns in a study conducted by Dr. Patricia N. R. Pereira.

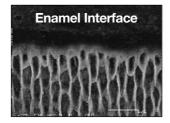

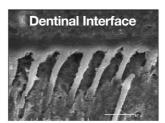

SEM images reveal excellent bond adaptation to etched enamel and dentinal surfaces.

DR. PATRICIA N. R. PEREIRA

Adper Scotchbond 1 XT Adhesive offers greater confidence.

- Superior bond strength provides exceptional bonding performance.
- Nanofiller is stable and will not settle – no shaking required.
- Compatible with all light-cured composite restoratives available today.

^{*}With 3M™ ESPE™ RelyX™ Ceramic Primer and 3M™ ESPE™ RelyX™ Veneer Cement.
**When combined with 3M™ ESPE™ RelyX™ARC Adhesive Resin Cement.

Adper[™] Scotchbond[™] 1 XT Adhesive from 3M ESPE offers improved performance.

Adper Scotchbond 1 XT adhesive is based on the clinically-proven Adper $^{\text{\tiny TM}}$ Scotchbond $^{\text{\tiny TM}}$ 1 Adhesive. It is a total-etch, visible-light activated dental bonding agent – now incorporating nanofiller technology.

In addition to higher bond strengths, there are more good reasons to use Adper Scotchbond 1 XT adhesive:

- Two popular delivery options: unit dose system or vial.
- Save time with the easy-to-use 3M ESPE unit dose. Promotes hygienic practices.
- Get consistency in every drop using the vial delivery. 3M ESPE nanofiller won't settle, so dispenses perfectly mixed straight from the vial without shaking before use.
- It bonds to etched enamel and dentinal surfaces, so it's excellent for all light-cured direct and indirect indications.

The total-etch adhesive you have trusted since 1997 now includes nanofiller for even better performance.

And, dentists now can choose a vial or unit dose delivery system!



Product Information

Adper™ Scotchbond™ 1 XT Unit Dose

51010 Adper Scotchbond 1 XT

Unit Dose Kit (40)

Kit contains: 40 Adper Scotchbond 1 XT Adhesive Unit Dose Applicators¹, 1 – 3ml Etchant Syringe¹.

51011 Adper Scotchbond 1 XT Adhesive

Unit Dose Kit (100)

Kit contains: 100 Adper Scotchbond 1 XT Adhesive Unit Dose Applicators¹, 1 – 3ml Etchant Syringe¹.



Adper™ Scotchbond™ 1 XT Vial

4241 Adper Scotchbond 1 XT Adhesive

Intro Kit (vial)

Kit contains: 1 Adper Scotchbond 1 XT Vial - 6g Adhesive¹, 2 - 3ml Etchant Syringes¹, 1 Transcoject Tip¹, Mixing Wells¹, 1 Brush Handle¹, 1 Bag of Brush Tips1.

4242 Adper Scotchbond 1 XT Refill

1 Adper Scotchbond 1 XT Vial - 6g Adhesive.

¹Also available separately.





Name

Vskd Asklkfgajer Fayhin Sieklt Hainety Asklk Sieklt Hainety Country

Name

Vskd Asklkfgajer Fayhin Sieklt Hainety Asklk Sieklt Hainety Country

Name

Vskd Asklkfgajer Fayhin Sieklt Hainety Asklk Sieklt Hainety Country

3M, ESPE, Adper, Scotchbond are trademarks of 3M or 3M ESPE AG. Used under license in Canada. OptiBond Solo is a brand of Kerr Dental.

0000000000/0 (xx.xx)



waste paper 10% post-consumer

Printed in Country © 3M 2004