

**3M** ESPE

Adhesive performance.  
Etching, priming and bonding  
in seconds.



**90**  
million  
applications

# Adper Prompt L-Pop<sup>TM</sup>

*Self-Etch Adhesive.*

# Adper™ Prompt™ L-Pop™ from 3M ESPE. Making working with composites and compomers easier.

## A quick, secure hold.

90 million successful clinical applications are all the proof you need: Adper™ Prompt™ L-Pop™ makes it particularly easy to achieve safe adhesion of light-hardening composite and compomer fillings. As a development of Prompt™ L-Pop™, Prompt™ L-Pop™ is an innovative solution for all who wish to save time on application without compromising on optimum adhesion.

## Quick, simple, safe – this is how it works:

### Simple activation.

Press – fold – press – ready! The adhesive is ready to apply in seconds.



Press ...



fold ...



press ...



ready!

### Save valuable time in everyday practice.

Simply rub in for 15 seconds and then blow dry. Apply a second layer, blow dry and light for 10 seconds\* (with LEDs, halogen plasma or laser light).



Not activated

### Adhesion improved!

Prompt™ L-Pop™ already had excellent adhesion to enamel and dentine – even without a separate etching stage. With Adper™ Prompt™ L-Pop™, we have improved dentine adhesion still further.

### Ensure better hygiene during treatment.

Adper™ Prompt™ L-Pop™ means no dirt, no spills and no contamination.

### Always mixed up fresh – for optimum results.

Adper™ Prompt™ L-Pop™ is mixed fresh for every application – the innovative application method makes it easy.

### Everything under control, all the time.

You can see whether the L-Pop system is activated by the yellow, central pad, which is pushed outwards when you press the outer, top pad (see picture). The yellow colour of the mixed adhesive also allows you to check the process visually.

### Make better use of cavity walls.

Compared to Prompt™ L-Pop™, Adper™ Prompt™ L-Pop™ is more viscous, which means better coverage.

**Summary: Make the most of the benefits of 5 years of practical experience and use Prompt™ L-Pop™ successfully in your practice!**

\* Depending on curing method




Activated

# Scientific research proves: It works!

## Prompt™ L-Pop™ produced excellent clinical results for over three years:

The Prompt™ L-Pop™ study demonstrated that the L-Pop™ system works. The method: 25 class III and class V fillings were put in for 17 patients. 20 fillings were clinically evaluated within 36 months. The results are given as percentages, using modified USPHS criteria. (C. Munoz et al., IADR 2004; Abstract 0541)

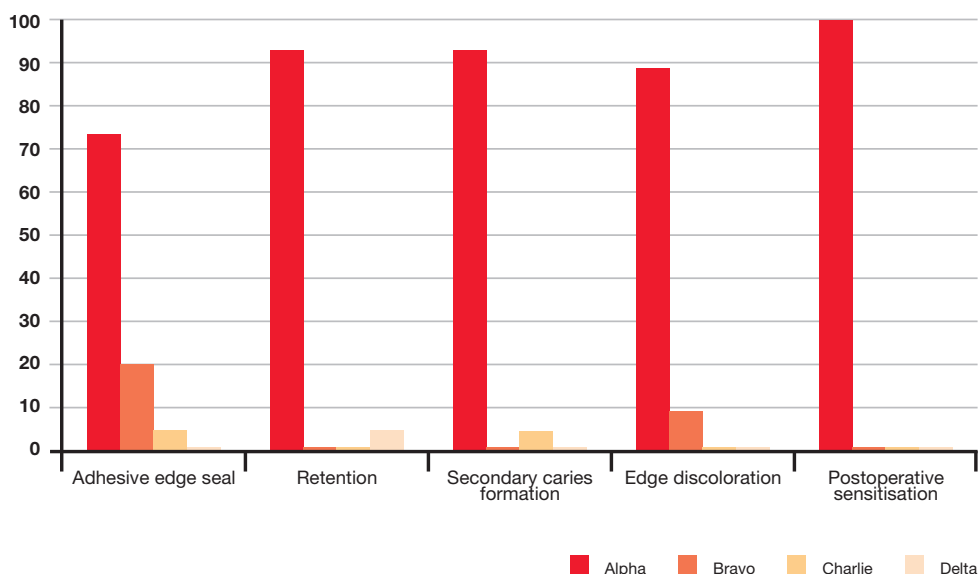


### Conclusions:

Results after 3 years

- ✓ Good edge seal.
- ✓ High retention.
- ✓ Very low rate of secondary caries formation.
- ✓ Low level of edge discoloration.
- ✓ Practically no postoperative sensitisation.


### Adper™ Prompt™ L-Pop™

- Successfully in use for 4 years.
- Tried and tested in clinical practice over 90 million times.
- Excellent edge quality.
- Very low rate of postoperative sensitivity.
- Interested in more scientific results? please order: A collection of Scientific Results

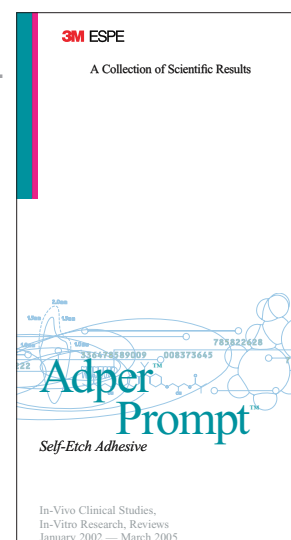

# Adper™ Prompt™ L-Pop™ – etching, priming and bonding in seconds.

## Step by step instructions:

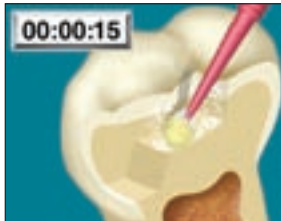

- 1 Apply the adhesive across the whole surface of the cavity. Rub it in for 15 seconds.



- 2 Dry the adhesive. This forms a thin film.



- 3 Now apply a second coat of adhesive.



- 4 Dry the adhesive again. This forms a thin film again.



- 5 Then allow to harden in light for 10 seconds (3 seconds in plasma light).

**3M ESPE**

3M ESPE AG · ESPE Platz  
82229 Seefeld · Germany  
E-Mail: [info3mespe@mmm.com](mailto:info3mespe@mmm.com)  
Internet: <http://www.3mespe.com>

3M, ESPE, Adper, Prompt and L-Pop are trademarks of 3M or 3M ESPE AG.

04.2006

Prime & Bond NT is a trademark of Caulk/Dentsply.

©3M 2006