

Protemp™ Crown Temporization Material Indirect Technique



Molar



Bicuspid
(Premolar)




Cuspid
(Canine)




1




2

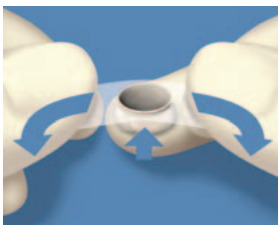

3

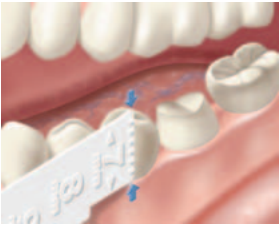


4

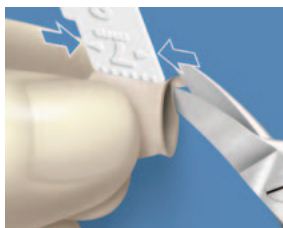

5



6

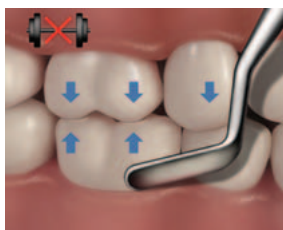

7



8



9



10




11



12




13

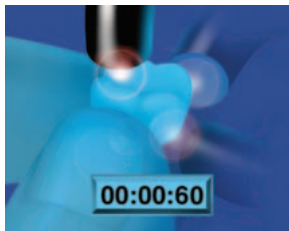


14




15

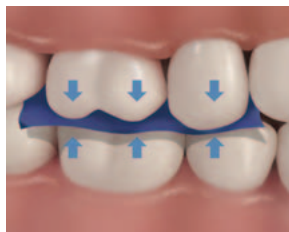


16

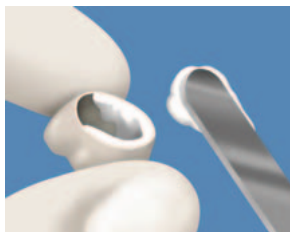

17

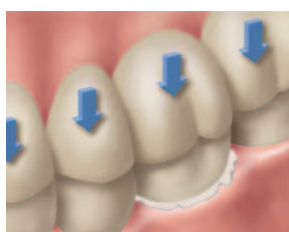


18

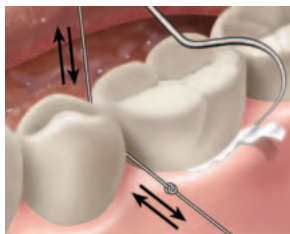

19

Size Selection Chart

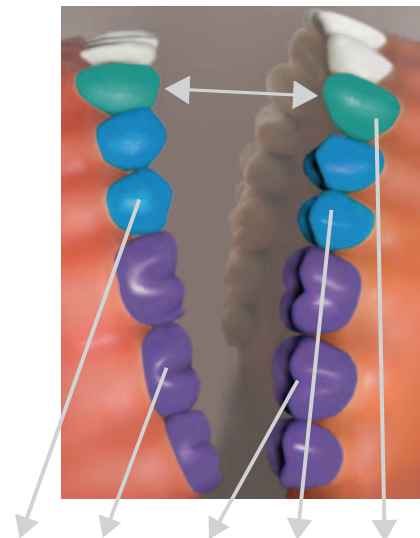

Crown Name	Approximate Range Mesial Distal Width*
CUSPID (Canine) Universal	
Cuspid (Canine) Universal S	6-7 mm
Cuspid (Canine) Universal L	8-9 mm
BICUSPID (Premolar) Upper	
Bicuspid (Premolar) Upper S	6-7 mm
Bicuspid (Premolar) Upper L	7-8 mm
MOLAR (1st & 2nd) Upper	
Molar Upper S	9-10 mm
Molar Upper L	11-12 mm
MOLAR (1st & 2nd) Lower	
Molar Lower S	9-10 mm
Molar Lower L	11-12 mm
BICUSPID (Premolar) Lower	
Bicuspid (Premolar) Lower	6-9 mm

* The wedging point relative to the measuring tool identifies the approximate mesial-distal width and size range for selecting a Protemp Crown