


## Filtek™ Supreme Ultra Ultra Universal Restorative



Select shade using the VITAPAN® classical shade guide and the 3M™ ESPE™ Filtek™ Supreme Ultra Universal Restorative Shade Selector Wheel.

- Prepare the tooth and isolate.

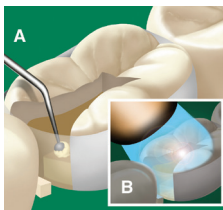

1

### Apply liner/base if desired:


- Mix 3M™ ESPE™ Vitrebond™ or 3M™ ESPE™ Vitrebond™ Plus.
- Apply a thin layer of the liner/base to dentin surfaces.
- Light-cure.



2

### Bonding procedure:

- Follow the bonding procedure provided in the instructions for use of the adhesive system of choice.

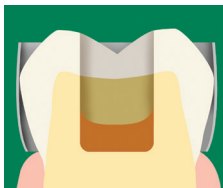

3

### Place composite – proximal box:

- To ensure good adaptation, place chosen **Body** shade of Filtek Supreme Ultra Restorative in a 1mm increment in the proximal box and on the floor of the cavity.
- Light cure.

## Filtek™ Supreme Ultra


### Ultra Universal Restorative

#### All halogen and LED lights (with output 400-1000 mW/cm<sup>2</sup>)

Shades	Increment	Cure time
Body, Enamel, Translucent	2.0 mm	20 sec.
Dentin, A6B and B5B	1.5 mm	40 sec.

#### Elipar™ LED Curing Lights (with output 1000-2000 mW/cm<sup>2</sup>)

Shades	Increment	Cure time
Body, Enamel, Translucent	2.0 mm	10 sec.
Dentin, A6B and B5B	1.5 mm	20 sec.

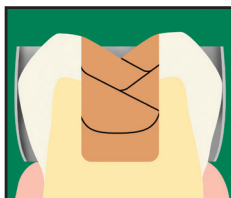


#### Place composite:

- Continue placing chosen **Body** shade of Filtek Supreme Ultra restorative in increments.
- Use remaining coronal tooth structure as a guide for placement and adaptation.

*Overfill slightly.*

- Light cure.

## Filtek™ Supreme Ultra Ultra Universal Restorative

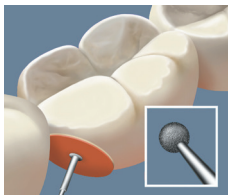

5

### Finish and Polish:

- Finish occlusal surface using an appropriate finishing instrument.
- Finish interproximal surfaces with 3M™ ESPE™ Sof-Lex™ Extra Thin Discs and 3M™ ESPE™ Sof-Lex™ Strips.



6

### Occlusion:

- Check lateral and centric occlusion.
- Adjust as necessary.

### 3M ESPE

Dental Products, 2510 Conway Avenue, St. Paul, MN 55144-1000 USA

### 3M ESPE Customer Care/MSDS Information:

**U.S.A. 1-800-634-2249 and Canada 1-888-363-3685**

VITAPAN is a trademark of VITA Zahnfabrik H. Rauter GmbH & Co.KG

3M, ESPE, Filtek, Vitrebond and Sof-Lex are trademarks of 3M or 3M Deutschland GmbH.

© 3M 2015. All rights reserved.



44-0050-7101-2-A