

## Filtek™ Supreme Ultra Ultra Universal Restorative



1

### Preparation:

- Select tooth shade using the VITAPAN® classical shade guide.
- Select composite shades using the 3M™ ESPE™ Filtek™ Supreme Ultra Universal Restorative Shade Selector as a guide.
- Prepare the tooth.
- Bevel enamel margins.



2

### Bonding procedure:

- Follow the bonding procedure provided in the instructions for use of the adhesive system of choice.



3

### Place composite – dentin replacement:

- Place and cure **Dentin** shade of Filtek™ Supreme Ultra Restorative to provide opacity and color intensity.
- Light cure.

#### All halogen and LED lights (with output 400-1000 mW/cm<sup>2</sup>)

Shades	Increment	Cure time
Body, Enamel, Translucent	2.0 mm	20 sec.
Dentin, A6B and B5B	1.5 mm	40 sec.

#### Elipar™ LED Curing Lights (with output 1000-2000 mW/cm<sup>2</sup>)

Shades	Increment	Cure time
Body, Enamel, Translucent	2.0 mm	10 sec.
Dentin, A6B and B5B	1.5 mm	20 sec.

## Filtek™ Supreme Ultra Ultra Universal Restorative



4

### Place composite – dentin replacement:


- Place and cure **Body** shade of Filtek Supreme Ultra to the dentin enamel junction.



5

### Place composite – enamel replacement:

- Fill remaining preparation using **Enamel** and/or **Translucent** shades of Filtek Supreme Ultra restorative to provide translucency for depth of color and incisal edge match.
- Light cure.

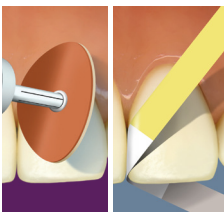

6

### Finish and Polish:

- Use 3M™ ESPE™ Sof-Lex™ Finishing and Polishing Discs and Strips.
- The aesthetic result will be enhanced by creating surface texture similar to the adjacent teeth.

### 3M ESPE

Dental Products, 2510 Conway Avenue, St. Paul, MN 55144-1000 USA

### 3M ESPE Customer Care/MSDS Information:

U.S.A. 1-800-634-2249 and Canada 1-888-363-3685

VITAPAN is a trademark of VITA Zahnfabrik H. Rauter GmbH & Co.KG

3M, ESPE, Filtek and Sof-Lex are trademarks of 3M or 3M Deutschland GmbH.

© 3M 2015. All rights reserved.



44-0007-7667-2-A