


Clarity™ SL and SmartClip™ SL3 Self-Ligating Brackets

Archwire Sequencing Overview – .022 Slot


As recommended by Dr. Anoop Sondhi

Archwire	Appointment Interval	Total Duration	Function
.012 or .014 Nitinol Super-Elastic	8 weeks	8 weeks	Preliminary alignment, leveling and rotation
.014 or .016 Nitinol Classic Tandem	8 weeks	8 weeks	Complete alignment, leveling and rotation
.017 × .025 Beta III Titanium	8-10 weeks	16-30 weeks	Provide torque control, arch consolidation, use of inter-arch elastics
.019 × .025 Beta III Titanium or Stainless Steel	8-10 weeks	16-30 weeks	
Retraction Arches	6-8 weeks	12-24 weeks	Space closure, retraction into extraction sites

Class I, Minor Crowding, Non Extraction

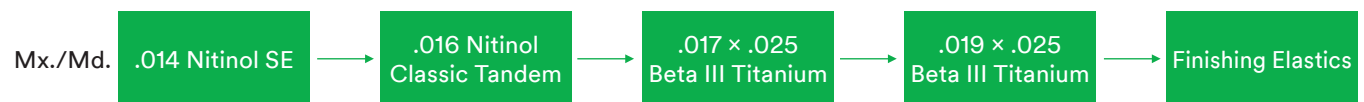

Class I, Moderate Crowding, Non Extraction

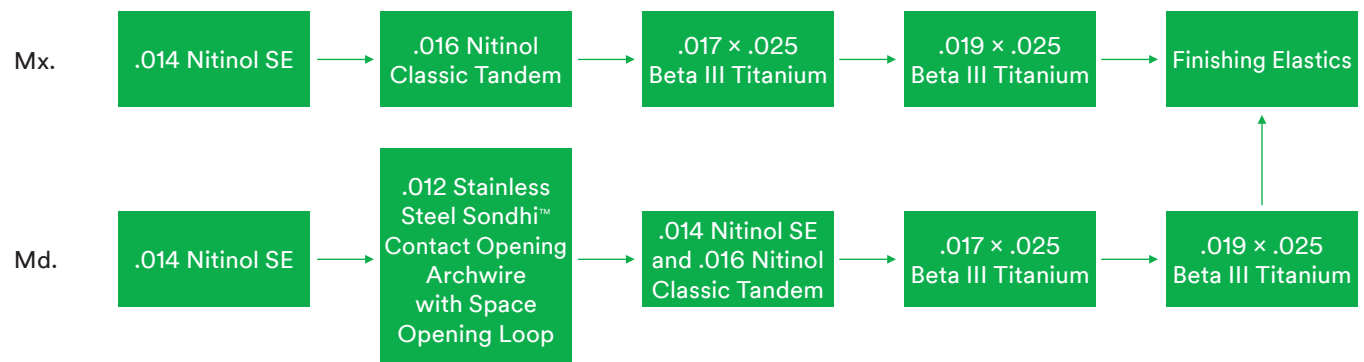


For details, please view the CE Class titled the “SmartClip™ and Clarity™ SL Self-Ligating Treatment Systems: Recommended Archwire Sequences” on 3MUnitekTraining.com.

Clarity™ SL and SmartClip™ SL3 Self-Ligating Brackets


Archwire Sequencing Overview – .022 Slot

As recommended by Dr. Anoop Sondhi


Class II, Non Extraction

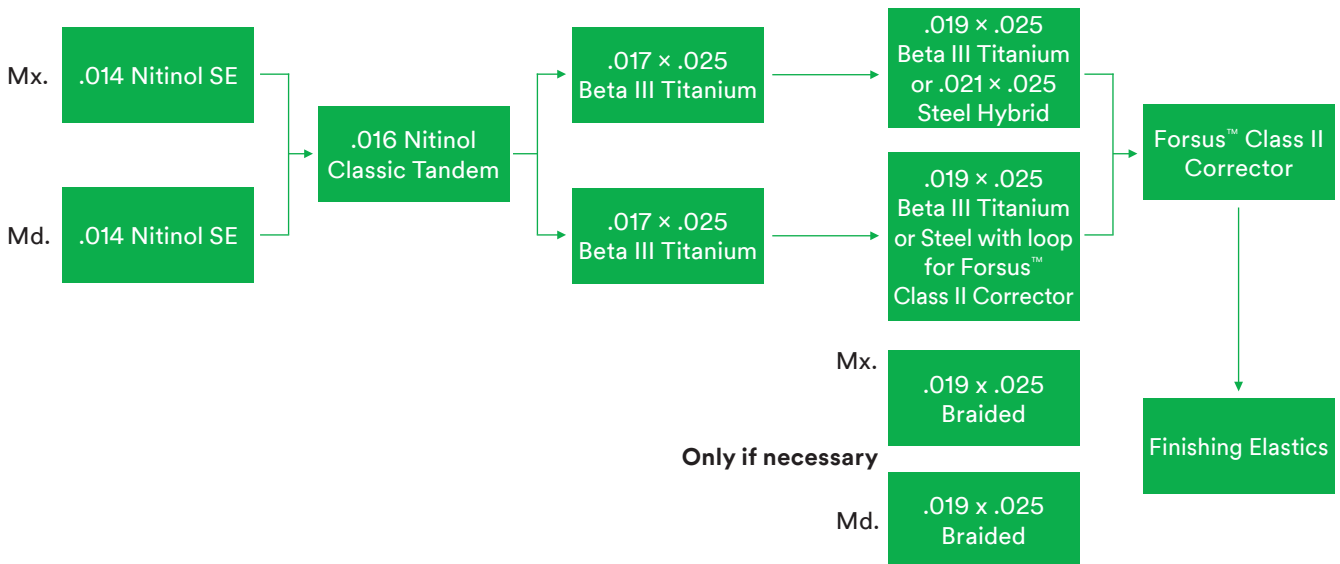

Extraction Arch, Maximum Anchorage

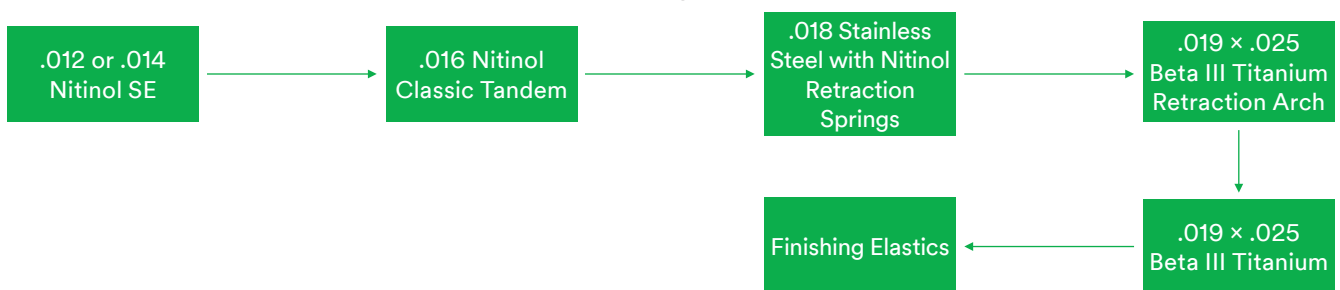

Extraction Arch, Minimum Anchorage

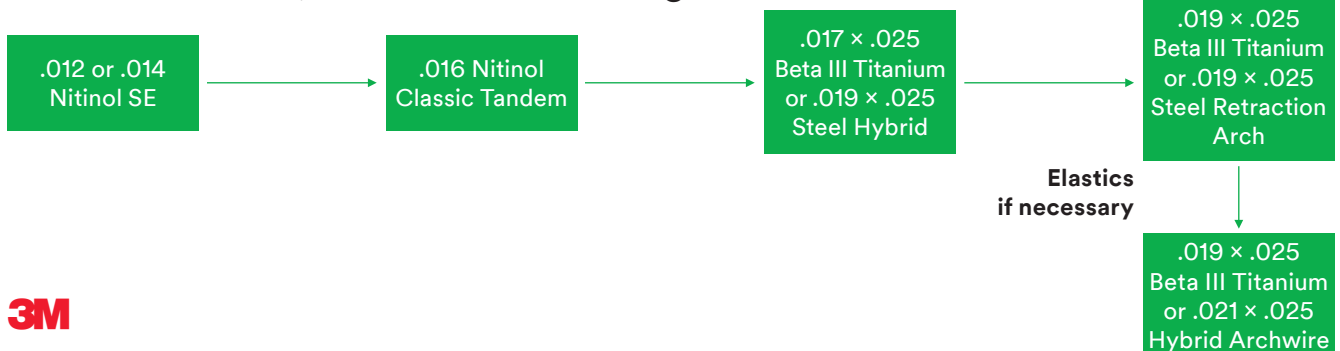

3M Oral Care
2510 Conway Avenue
St. Paul, MN 55144-1000 USA

Phone 1-800-423-4588
Web 3M.com/ortho

3M Canada
Health Care Division
300 Tartan Dr.
London, ON N5V 4M9 Canada
Phone 1-800-443-1661

3M, Clarity, Forsus, SmartClip,
and Sondhi are trademarks of 3M.
Used under license in Canada.
© 3M 2016. All rights reserved.
70-2021-5247-9 1601