

3M™ Steri-Strip™ C1544 Compound Benzoin Tincture

Applicator Exterior Sterile

Product Description

3M™ Steri-Strip™ Compound Benzoin Tincture is an alcoholic solution of compound benzoin.

Indications for Use

Steri-Strip Compound Benzoin Tincture is intended for use as an adhesive adjunct, when enhancement of surgical tape and/or adhesive wound closure adhesion is desired.

Contraindications

Steri-Strip Compound Benzoin Tincture should not be used on patients with previous history of sensitivity to compound benzoin tincture.

Warnings

1. Allergic contact dermatitis to compound benzoin tincture has been reported.
2. WARNING flammable liquid and vapor.
3. Compound Benzoin Tincture is flammable until it has completely dried on the skin.
4. Compound Benzoin Tincture should only be applied when no ignition sources, flames, or heat-producing devices are in use.
5. For external use only.

Precautions

Spillage of compound benzoin tincture into the wound should be avoided, as it may impair the wound healing process.

Directions for Use

1. Clean dry skin for at least 2 ½ inches (6,4cm) around the wound making sure to remove skin oil or exudate. For best results, use a sponge moistened with alcohol or saline followed by a firm wipe with a dry sponge.
2. Using aseptic technique, peel back lid and deliver vial.
3. Gently squeeze vial to crush inner ampoule, thereby releasing compound benzoin tincture. Do not roll crushed ampoule between fingers as this may cause glass to penetrate outer vial.
4. Carefully apply a thin film of compound benzoin tincture to the skin on each side of the wound with strokes parallel to the wound. Begin at the wound edges and work outward. Avoid contact with the wound.
5. Allow tincture to dry before applying surgical tapes and/or adhesive wound closure bandages (approximately 30-45 seconds).

Storage/ Shelf Life/Disposal

Store below 40°C (104°F). Avoid excessive heat or humidity.

How supplied

Applicator sterile unless package is opened or damaged.

For further information, please contact your local 3M representative or contact us at 3M.com and select your country.

Explanation of Symbols



Caution, see instructions for use



Do not reuse



Manufacturer



Date of manufacture



Sterilized using ethylene oxide



Temperature limitation (40C/104F)



WARNING! FLAMMABLE LIQUID AND VAPOR!

CE 0086

Made in U.S.A. with globally sourced materials for

3M Health Care

2510 Conway Ave.

St. Paul, MN 55144 U.S.A.

1-800-228-3957

3M.com/WoundCare

© 2018, 3M. All rights reserved.

3M and Steri-Strip are trademarks of 3M.

Issue Date: 2018-03
34-8722-7224-9

EC REP **3M Deutschland GmbH**
Health Care Business
Carl-Schurz-Str. 1
41453 Neuss
Germany