

## Nano Hybrid Universal Restorative




### Preparation

- Select tooth shade using the VITAPAN® classical shade guide.
- Prepare tooth using conventional cavity preparations.



### Bonding procedure

- Follow the bonding procedure provided in the instructions for use of the adhesive system of choice.

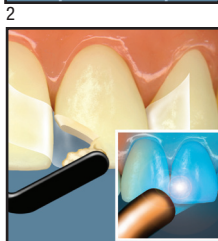

### Place composite

- Place composite in increments or layers.
- Light cure.


3

**Nano Hybrid Universal Restorative****Minimum intensity 400 mW/cm<sup>2</sup> lights**

| Shades                                      | Incremental Depth | Cure Time |
|---|-------------------|-----------|
| A1, A2, A3, A3.5, A4,<br>B1, B2, B3, C2, D3 | 2.0 mm            | 20 sec.   |
| OA2, OA3                                    | 1.5 mm            | 20 sec.   |

**Minimum intensity 1000 mW/cm<sup>2</sup> lights**

| Shades                                      | Incremental Depth | Cure Time |
|---|-------------------|-----------|
| A1, A2, A3, A3.5, A4,<br>B1, B2, B3, C2, D3 | 2.0 mm            | 10 sec.   |
| OA2, OA3                                    | 1.5 mm            | 10 sec.   |

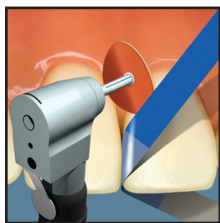**Finish and polish the restoration**

5

**3M ESPE**Dental Products, St. Paul, MN 55144-1000 USA • [www.3MESPE.com](http://www.3MESPE.com)**3M ESPE Customer Care/MSDS Information:****U.S.A. 1-800-634-2249 and Canada 1-888-363-3685**VITAPAN® is a registered trademark of VITA Zahnfabrik  
3M, ESPE and Paradigm are trademarks of 3M or 3M ESPE AG.

© 3M 2011. All rights reserved.