




## Nano Hybrid Universal Restorative

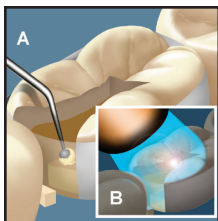

Apply liner/base if desired

**Bonding procedure**

- Follow the bonding procedure provided in the instructions for use of the adhesive system of choice.

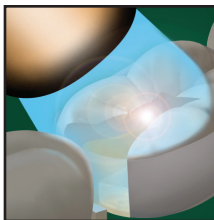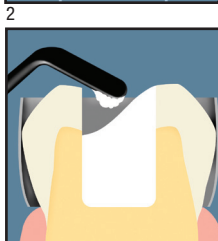**Place composite**

- Place composite in increments or layers.
- Light cure.


## Nano Hybrid Universal Restorative

### Minimum intensity 400 mW/cm<sup>2</sup> lights

Shades	Incremental Depth	Cure Time
A1, A2, A3, A3.5, A4, B1, B2, B3, C2, D3	2.0 mm	20 sec.
OA2, OA3	1.5 mm	20 sec.

### Minimum intensity 1000 mW/cm<sup>2</sup> lights

Shades	Incremental Depth	Cure Time
A1, A2, A3, A3.5, A4, B1, B2, B3, C2, D3	2.0 mm	10 sec.
OA2, OA3	1.5 mm	10 sec.

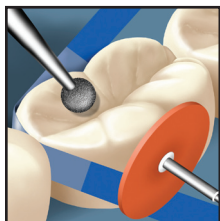

### Finish and polish the restoration

4



### Occlusion

- Check lateral and centric occlusion.
- Adjust as necessary.

5

### 3M ESPE

Dental Products, St. Paul, MN 55144-1000 USA • [www.3MESPE.com](http://www.3MESPE.com)

### 3M ESPE Customer Care/MSDS Information:

**U.S.A. 1-800-634-2249 and Canada 1-888-363-3685**

3M, ESPE and Paradigm are trademarks of 3M or 3M ESPE AG.

© 3M 2011. All rights reserved.