

RelyX™ Luting Plus Automix

Resin Modified Glass Ionomer Cement



Now in a self-mixing
syringe delivery system

The convenient Automix delivery system makes it easier to use.

RelyX™ Luting Plus Automix Resin Modified Glass Ionomer Cement


Using the dentist-preferred automix delivery system is more convenient than ever.

The RelyX™ Luting Plus Automix Cement puts an end to loading, measuring and hand mixing. The Automix tip design self-mixes the two pastes as they are being dispensed. The result is an easier and more precise delivery.


The lightweight Automix syringe is sized for easy handling and provides flexible dosing so the right amount can be dispensed for each case. The cement has an ideal consistency so it loads into the restoration and allows for easy seating.

The Automix system delivers precise results ... in fewer steps.

Dispenses directly into the restoration

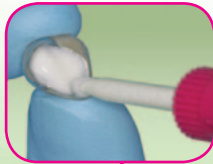

Automix tip mixes the two pastes consistently every time

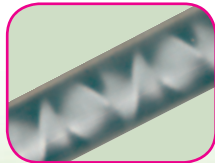

Improved paste/paste with new tack light cure option



Summary of advantages

- Easier to use and preferred over other RMGI automix systems
- Automix delivery allows for direct delivery into the crown (no hand mixing)
- Sustained fluoride release for caries-prone patients
- Virtually no post-operative sensitivity

The only RMGI cement with a tack light cure option.

Enjoy faster cleanup of excess cement!

You can speed up the gel state by light curing excess RelyX Luting Plus Automix cement for about 5 seconds per surface with a conventional light-curing device. Or, you can wait for the material to self-cure—about 2 minutes after seating in the mouth.


Once excess RelyX Luting Plus cement reaches a gel state, it can be easily dislodged in large sections from the margins with a suitable instrument.



Proven performance makes it easier to choose.

Proven to have better bond strengths.

The dispenser may be new, but you can expect the same reliable results proven by RelyX Luting Plus Cement in the Clicker™ Dispenser. RelyX Luting Plus Automix cement provides molecular bonding to the tooth surface without the use of a separate tooth conditioner. And, there's virtually no post-operative sensitivity. Plus, it offers sustained fluoride release to benefit your caries-prone patients. It's the reliable luting cement that's ideal for everyday use.

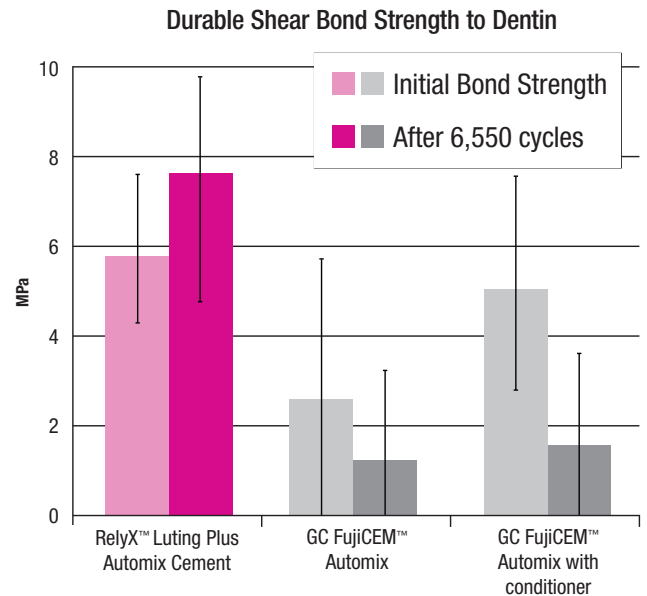

A study on the effect that thermal cycling has on adhesion to bovine dentin for various RMGI cements, proved RelyX™ Luting Plus Automix Cement provides better bond strength compared to other leading cements—**both with and without an added conditioner.**

Wang, et al. AADR/IADR 2011, #3196.

Proven by the countless restorations placed every day.

It's the trusted cement for a wide range of indications including:

- Porcelain-fused-to-metal (PFM) crowns and bridges
- Metal crowns, inlays and onlays
- All-alumina or all-zirconia strengthened core ceramic restorations (such as Lava™ or Procera®)
- Orthodontic bands and appliances
- Final cementation of PFMs, metal crowns, all-alumina or all-zirconia strengthened core ceramic restorations to implant abutments



RelyX™ Cements. Easy to choose. Easy to use.

There's a proven RelyX™ solution for your every cementation need.



Recommended Indications*

Metal/PFMs
Porcelain/Ceramic
Veneers
Zirconia
CAD/CAM
Ceramic and Composite
Posts
Pediatric Crowns
Metal and Composite
Maryland Bridges
Metal and Composite

RelyX™ Unicem and RelyX™ Unicem 2
Self-Adhesive Resin Cement



RelyX™ Luting Plus Automix Cement,
RelyX™ Luting Plus Cement




RelyX™ Veneer Cement



*These are only recommended indications. For a complete list, refer to the respective product Instructions for Use.

Delivery Method

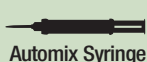

Ordering Information



RelyX™ Luting Plus Automix Resin Modified Glass Ionomer Cement—Trial Kit (3535TK)



RelyX™ Luting Plus Automix Resin Modified Glass Ionomer Cement—Value Pack (3535)

Item #	Product Information
3535TK	RelyX™ Luting Plus Automix Resin Modified Glass Ionomer Cement—Trial Kit Kit includes: 1 – 8.5g Automic syringe; 12 – mixing tips; Technique Guide; Instructions for Use
3535	RelyX™ Luting Plus Automix Resin Modified Glass Ionomer Cement—Value Pack Includes: 3 – 8.5g Automic syringes; 36 – mixing tips; Technique Guide; Instructions for Use
3540	RelyX™ Luting Plus Automix Resin Modified Glass Ionomer Cement—Automic Tips Includes: 12 – mixing tips

Customer Care Center: 1-800-634-2249

www.3MESPE.com/RelyX

3M ESPE

Dental Products

3M Center
Building 275-2SE-03
St. Paul, MN 55144-1000
U.S.A.
1-800-634-2249

3M Canada

Post Office Box 5757
London, Ontario N6A 4T1
Canada
1-888-363-3685

Please recycle. Printed in U.S.A.
© 3M 2011. All rights reserved.
70-2013-0487-3

3M, ESPE, Clicker, Lava and RelyX are trademarks of 3M or 3M ESPE AG. Used under license in Canada. GC FujiCEM and Procera are not trademarks of 3M.