

Scotchbond™ Universal Adhesive



Indication

- Direct restorations with light curing composites and compomers
- Intraoral repair using light curing composites
- Root surface desensitization
- Sealing of cavities before filling with amalgam
- Sealing of cavities and tooth preparations before temporary placement of indirect restorations



1


- Apply the adhesive to the prepared tooth and rub it in for 20 seconds.



2

- Gently air dry the adhesive for approximately 5 seconds to evaporate the solvent.

Scotchbond™ Universal Adhesive

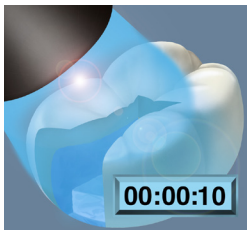

- Light cure for 10 seconds.

3

3M ESPE Customer Care/MSDS Information:

U.S.A. 1-800-634-2249 and Canada 1-888-363-3685

3M, ESPE and Scotchbond™ are trademarks of 3M or 3M Deutschland GmbH.

Used under license in Canada. © 3M 2012. All rights reserved.

3M ESPE

Dental Products, 2510 Conway Avenue, St. Paul, MN 55144-1000 USA • www.3MESPE.com



44-0007-7255-6-A