


## Scotchbond™ Universal Adhesive



### Indication

- Bonding of dual cure cements and core build-up materials
- Bonding of self-cure composites

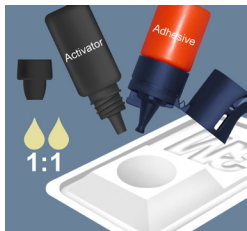

1

- Place one drop each of Scotchbond™ Universal Adhesive and dual cure activator\* in a mixing well and mix for 5 seconds.

\* The Scotchbond™ Universal Dual Cure Activator is not required when using RelyX™ Ultimate Cement. When using RelyX™ Ultimate, please proceed to Step #2.



2


- Apply the adhesive or adhesive mixture to the prepared tooth and rub it in for 20 seconds.



3

- Gently air dry the adhesive for approximately 5 seconds to evaporate the solvent.

## Scotchbond™ Universal Adhesive

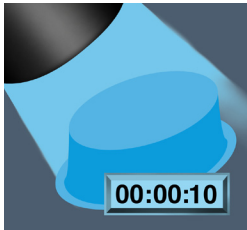

- Light cure for 10 seconds. (optional step with RelyX™ Ultimate Cement)

4



- Apply the adhesive or adhesive mixture to the restoration for 20 seconds.

5



- Air dry for 5 seconds.  
**Do not light cure!**  
Proceed with cementing the restoration according to manufacturer's instructions.

6

**Scotchbond™ Universal**  
Adhesive

---

**Scotchbond™ Universal**  
Adhesive

---

**3M ESPE Customer Care/MSDS Information:****U.S.A. 1-800-634-2249 and Canada 1-888-363-3685**

3M, ESPE, Scotchbond™ and RelyX™ are trademarks of 3M or 3M Deutschland GmbH.  
Used under license in Canada. © 3M 2012. All rights reserved.

**3M ESPE**

Dental Products, 2510 Conway Avenue, St. Paul, MN 55144-1000 USA • [www.3MESPE.com](http://www.3MESPE.com)



44-0007-7256-4-A