

## Scotchbond™ Universal Adhesive



### Indication

- Light-cure sealing of fissures




1

- Apply the adhesive to the prepared tooth and rub it in for 20 seconds.



2


- Gently air dry the adhesive for approximately 5 seconds to evaporate the solvent.

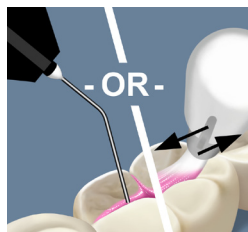

3

- Apply a light-cure fissure sealant.

## Scotchbond™ Universal Adhesive

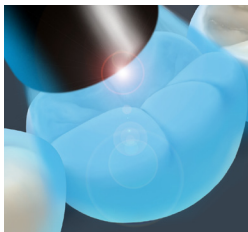

- Light cure the sealant and adhesive together according to the sealant manufacturer instructions for use.

4

### **3M ESPE Customer Care/MSDS Information:**

**U.S.A. 1-800-634-2249 and Canada 1-888-363-3685**

3M, ESPE and Scotchbond™ are trademarks of 3M or 3M Deutschland GmbH.

Used under license in Canada. © 3M 2012. All rights reserved.

### **3M ESPE**

Dental Products, 2510 Conway Avenue, St. Paul, MN 55144-1000 USA • [www.3MESPE.com](http://www.3MESPE.com)

