

## Scotchbond™ Universal Adhesive



### Indication

- Cementation of veneers using RelyX™ Veneer Cement or RelyX™ Ultimate Cement



1

- Apply a commonly used phosphoric acid etching gel (about 35%), e.g., Scotchbond™ Universal Etchant to the enamel and dentin for 15 seconds.



2

- Rinse thoroughly with water for 10 seconds.



3

- Dry with cotton pellets.

## Scotchbond™ Universal Adhesive



- Apply Scotchbond™ Universal Adhesive immediately to the entire surface of the enamel and dentin and rub it in for 20 seconds.

4



- Gently air dry the adhesive for approximately 5 seconds to evaporate the solvent.

5



- **Do not light cure!**

6

## Scotchbond™ Universal Adhesive



7

- Apply Scotchbond™ Universal Adhesive to the hydrofluoric acid etched veneer surface.



8

- Gently air dry the adhesive for approximately 5 seconds to evaporate the solvent.



9

- **Do not light cure!**

## Scotchbond™ Universal Adhesive





- Cement the veneer with RelyX™ Veneer Cement or RelyX™ Ultimate Cement according to the pertinent instructions for use.

10



- Seat the veneer in place using gentle pressure.
- Spot cure the veneer into place on the facial surface away from the margins using a small diameter light guide for 20 seconds to secure it in place.
- Remove the excess cement from the margins using a blunt instrument or dry brush.

11

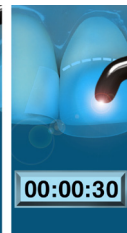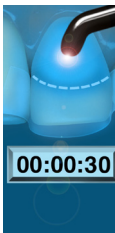

- Light cure the gingival edges first for 30 seconds, followed by light curing the middle and incisal edges each for 30 seconds.

12

**Scotchbond™ Universal**  
Adhesive

---

**Scotchbond™ Universal**  
Adhesive

---

**3M ESPE Customer Care/MSDS Information:****U.S.A. 1-800-634-2249 and Canada 1-888-363-3685**

3M, ESPE, Scotchbond™ and RelyX™ are trademarks of 3M or 3M Deutschland GmbH.  
Used under license in Canada. © 3M 2012. All rights reserved.

**3M ESPE**

Dental Products, 2510 Conway Avenue, St. Paul, MN 55144-1000 USA • [www.3MESPE.com](http://www.3MESPE.com)

