



Comfort Convenience Clinical Warming

Convenient hose ports for warming versatility

The 3M™ Bair Paws™+ gown incorporates a second clinical hose port in the lower portion of the gown, making clinician access simple by eliminating the need to reposition the gown when a procedure requires lower body warming.

As always, the Bair Paws system offers patient-adjustable warming with a simple turn of a dial. The Bair Paws warming unit's handheld controller allows patients to adjust the temperature of the air flowing through the gown to a temperature that's right for them. It is a perfect complement to a facility's patient satisfaction efforts.

- Flexibility for lower body warming needs.
- Two clinical hose ports eliminate the need to reposition the gown for many procedures.
- Both clinical ports include a hose port card, making use effortless.
- Pre-induction warming in the operating room is easy through the gown's lower clinical port.
- Patient-friendly post-operative clinical warming; the 3M™ Bair Hugger™ warming unit can be conveniently positioned for use in the lower clinical port.

3M™ Bair Paws™ + Gown Positioning

The 3M™ Bair Paws™ + gown provides flexible warming options. Two clinical hose ports make clinician access simple – just select the most convenient hose port for upper or lower body warming needs.



Bair Paws + gown on patient with right sleeve fasteners undone and gown folded away from surgical site. Upper Bair Hugger therapy hose port utilized.

Potential procedure: right shoulder

Bair Paws + gown removed from patient and laid across patient lengthwise. Upper 3M™ Bair Hugger™ therapy hose port utilized.



Potential procedure: hip

Bair Paws + gown on patient with lower portion of the gown folded up. Upper Bair Hugger therapy hose port utilized.



Potential procedures: knee, lower extremity, feet

Bair Paws + gown sleeve fasteners undone and gown slid down on the patient. Lower Bair Hugger therapy hose port utilized.

Potential procedures: shoulder, breast, neck





Bair Paws + gown sleeve fasteners undone and gown slid down on patient as patient is moved into lateral position. Lower Bair Hugger therapy hose port utilized.

Potential procedures:
thoracotomy, shoulder



Bair Paws + gown sleeve fasteners undone and gown slid down on patient as patient is moved into prone position. Lower Bair Hugger therapy hose port utilized.

Potential procedure: spinal



Bair Paws + gown on patient. Lower Bair Hugger therapy hose port utilized.

Potential procedures: eyes,
ENT, plastic facial


Bair Paws + gown removed from patient and laid across patient lengthwise. Upper Bair Hugger therapy hose port utilized.

Potential procedures:
urology, gynecology





3M™ Bair Paws™ Comfort Warming & Clinical Prewarming

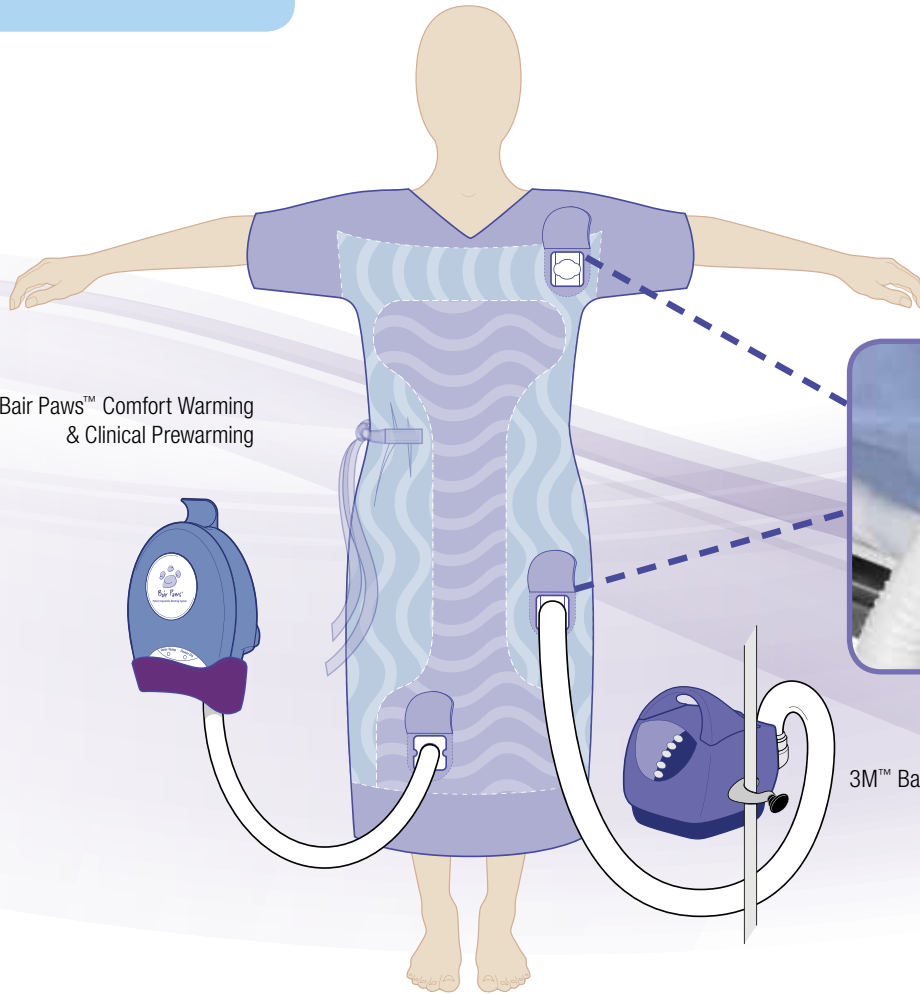

3M™ Bair Hugger™ Clinical Warming

Ordering Information

Product	Part Number	Sizes	Units
Patient Warming Gown	81102	Small (44" long; 55" sweep)	20/case
	81002	Standard (51" long; 64" sweep)	30/case
	81202	X-Large (51" long; 122" sweep)	20/case
Patient Warming Gown Kit	84102	Small (44" long; 55" sweep)	20/case
	84002	Standard (51" long; 64" sweep)	30/case
	84202	X-Large (51" long; 122" sweep)	20/case
Patient Warming Gown with Booties	83102	Small (44" long; 55" sweep)	20/case
	83002	Standard (51" long; 64" sweep)	30/case
	83202	X-Large (51" long; 122" sweep)	20/case
Patient Booties	90065	Standard Booties	30 pairs/case
	90091	XL Booties	30 pairs/case



Gown kit includes patient warming gown, bonnet, booties, personal belongings bag and shoe bag.



Arizant Healthcare Inc.,
a 3M company
 10393 West 70th St.
 Eden Prairie, MN 55344 USA
 Phone 800-733-7775
 Fax 800-775-0002
 www.bairpaws.com

3M is a trademark of 3M Company, used under license in Canada. BAIR HUGGER, BAIR PAWS and the BAIR PAWS logo are trademarks of Arizant Healthcare Inc., used under license in Canada. ©2012 Arizant Healthcare Inc. All rights reserved. 602964D 6/12

