

3M™ Microfoam™ Surgical Tapes and Sterile Tape Patch

Commonly Asked Questions

Question:

Is Microfoam tape hypoallergenic & natural rubber latex free?

Answer:

Yes.

Question:

When would you use Microfoam tape?

Answer:

Microfoam tape's distinguishing characteristics are: (1) occlusive/water resistant and (2) stretch in all directions. The most common Microfoam tape applications are for compression dressings (post hip surgery, after femoral sheath removal), dressings over joints (shoulders, knees), occlusive dressings (around chest tubes or to secure dressing after chest tube removal). If only stretch – not occlusiveness - is needed, 3M™ Medipore™/Medipore H tape(s) may be used for many of the same applications.

Question:

Is Microfoam tape sterile? Can it be sterilized?

Answer:

Microfoam tape is available in two forms: sterile patches and clean, but not sterile, rolls. Microfoam tape may be sterilized in a single layer by ethylene oxide but NOT by steam (autoclave). Although 3M provides these guidelines, it is up to the facility to ensure sterility. At this time, Sterrad™ processing with hydrogen peroxide is not recommended for Microfoam tape.

Question:

Is Microfoam tape the same as an “elastic” tape? What would be a cross-reference for an elastic tape?

Answer:

No, Microfoam tape does not contain elastic. Microfoam tape has the advantages of all directional stretch (across, bias/diagonal, and length-wise) rather than the two dimensional stretch (diagonal and length-wise) of elastic tapes, and Microfoam tape is natural rubber latex free. A 3M tape cross-reference for an elastic tape will depend on how the elastic tape is being used. Suggestions include:

- Around limbs (e.g., vein ligation) -> 3M™ Coban™ Self-Adherent Dressing
- For compression dressings -> Medipore or Medipore H tapes but note that the major direction of stretch is length-wise for most elastic tapes vs. cross-wise for the Medipore tapes. Microfoam tape is also appropriate for compression dressings, particularly if an occlusive tape is needed.
- Critical or heavy tubes -> 3M™ Cloth Adhesive Tape
- General dressings or tubing (epidural tubing) -> 3M™ Transpore™ White or Medipore/Medipore H tapes

Question:

Why do we have an expiration date for tapes? What happens when a tape “expires”?

Answer:

Although an expiration date is not required for surgical tapes in the U.S., an expiration date is required by a number of other countries. You might think of it as a “best quality if used by...”. In general, we do not expect changes in performance shortly after the expiration date, but 3M ensures performance within that time frame. The recommended shelf life for Microfoam tape rolls is 5 years under normal storage conditions.

Question:

What is a recommended method for applying a compression dressing with Microfoam tape?

Answer:

Because inappropriate strapping of tape may increase the risk of “tension blisters”, consider this approach:

1. Place the first section of tape without tension onto the skin on one side of the dressing, gently pressing downward.
2. With one hand, secure the tape on one edge of the dressing while slightly stretching the next section of the tape over the dressing.
3. Secure the tape on the opposite side of the dressing.
4. Press the remaining tape onto the skin without tension. Compression should come from the multiple layers of dressing material, not from excessive tension in the tape. Skin distention under the tape may result in blistering and should be relieved as soon as possible.

General Taping

Question:

Which of the surgical tapes are the most gentle?

Answer:

The most gentle tapes for repeated taping are 3M™ Medipore™ H, Micropore™, Micropore™ Plus, Medipore™, and Transpore™ White tapes.

Question:

Is there anything more to know than “sticky side down”?

Answer:

Yes, after choosing the appropriate tape, place the tape without tension onto the skin. In general, do not encircle a limb completely with tape since subsequent swelling or an infiltration may compromise circulation. The adhesives on the surgical tapes are “pressure sensitive”. This means that they are designed to adhere best when gentle, but firm, finger or hand pressure is applied to the tape, rubbing it into place. Skin has hills and valleys. Just laying a piece of tape on the skin only gives you contact with the top of the hills. Gentle, but firm strokes along the tape allows the adhesive to come into contact with more of the skin surface. This will increase initial adhesion and decrease the risk of “rolling” or losing your tape to friction. Usually, the tackier - or stickier - the tape, the less pressure is needed.

Question:

How can I increase tape adhesion?

Answer:**The key concepts to maximize adhesion are:**

- Start with clean, dry skin
- Touch the sticky surface of the tape as little as possible
- Apply sufficient pressure to the tape to get the adhesive into the nooks and crannies of the skin
- Cover adequate surface area so that the tape can support the tubing or dressing
- Obtain full contact between the tape and the skin or tubing so that moisture cannot slip between the two and loosen the tape

Most adhesives stick best to clean, dry surfaces so try to minimize moist conditions as much as possible.

- Remove substances that contain emollients or oils, such as most moisturizers and adhesive tape removers.
- If the skin is very oily, use a mild soap and water to remove the excess oil and pat dry. An alcohol wipe may also be used to remove the excess oil, but since it is very drying, it should be used with care
- To protect at-risk skin, 3M™ Cavilon™ No Sting Barrier Film may be used under the tape.
- If you are using a prep solution, let it dry completely before applying the tape.

Question:

How can I get tape to let go of my gloves?

Answer:

We tend to hold tape tightly, especially when tearing it off the roll and when gauging where we plan to put the tape. Because the tapes have pressure sensitive adhesives, when we hold them tightly, they stick even more to our gloves. But, the adhesives were designed to release from gloves so try holding the tape a little less tightly, and it will usually “let go”.

Question:

What is the best way to remove tape?

Answer:

Proper tape removal is critical in reducing the occurrence of traumatic skin injuries such as skin stripping. First, loosen the edges of the tape. You may “start” an edge of the tape by pressing a small piece of tape onto the corner, like a pop tab, and lifting – pressure sensitive adhesive! Stabilize the skin with one finger. Remove the tape “low and slow” in the direction of hair growth. Keep the tape close to the skin surface and pulled back over itself. Removing tape at an angle will increase tension on the epidermis and increase the risk of mechanical trauma. As the tape is removed, continue to support the newly exposed skin. Support close to the “peel edge” is particularly important for thin or easily distensible skin.

For tape that is strongly adhered to skin or hair, you may consider using a medical grade adhesive remover or moisturizer (moisturizer) to soften the adhesive. 3M tape adhesives are not readily dissolved in alcohol.

Hair presents special challenges. It can be difficult to obtain good initial adhesion over hair and yet, difficult to remove tape from hair after a time. Hair tends to grow medial to lateral, sometimes downward. A technique that may be helpful in removing tape from hair is to catch the upper, medial edge of the tape. Peel the edge of the tape back, forming a small triangle. Supporting the skin adjacent to the tape, glide a small amount of moisturizer on the leading (peel) edge of the tape. This is often enough to soften the adhesive and release it from hair. Continue to remove the tape “low and slow, back over itself” while gliding the moisturizer along the edge.

Question:

What if I used a moisturizer (cream) or an adhesive remover and need to tape over the same area?

Answer:

Reassess the skin. If you wish to retape over an area where you used adhesive remover or moisturizer, you will either need to remove the moisturizer or adhesive remover or use a moisturizer that allows you to tape over it. Some moisturizers (creams) that allow you to retape may actually increase adhesion of some tapes on some people, so it is important to use good technique for subsequent tape removal. You may wish to protect areas that will be exposed to repeated taping with an alcohol-free barrier film such as Cavilon No Sting Barrier Film. Removing the tape will also remove the barrier film, so the film should be reapplied and allowed to dry between tapings.

Question:

Where can I find the expiration date?

Answer:

The expiration date is embossed on the end of the tape box following the hourglass symbol. At this time, the lot number and expiration date cannot be printed inside the rolls because of limitations in space and printers.

Question:

Are Material Safety Data Sheets (MSDSs) required for surgical tapes?

Answer:

No. By definition, surgical tapes are “articles”, so MSDSs are not required.



Health Care

3M Center, Building 275-4W-02

St. Paul, MN 55144-1000

USA

1 800 228-3957

www.3M.com/healthcare

For More Information

Contact your 3M Health Care Sales Representative, or call the 3M Health Care Customer Helpline at **1-800-228-3957**.

These products can be ordered from your local distributor.

Outside the United States, contact the local 3M subsidiary.