

3M™ Medical Tapes
A Quick Reference for Choosing Tape



Because **Tape**
Matters



3M™ Medical Tapes Which Tape Should I Use?



“My 1st priority is at-risk skin.”



3M™ Kind Removal Silicone Tape

- Gentle tape adheres well, yet removes without disrupting fragile skin layers
- Reliable and consistent securement
- Easy to reposition and tear by hand
- Ideal for patients with sensitive, compromised or at-risk skin



“I need a tape for everyday use.”



3M™ Transpore™ White Tape

- Gentle tape is perforated for easy, straight, bi-directional tearing
- Good initial and long-term adhesion
- Easy to handle with gloves
- Ideal for securing dressings to dry or damp skin, taping fragile skin and anchoring tubing and devices



3M™ Medipore™ H Soft Cloth Tape

- High strength adhesive ensures reliable securement for both moist and dry skin
- Porous, soft cloth fabric allows skin to breathe, helping maintain skin integrity
- Stretches across and diagonally to accommodate swelling, distension and movement
- Easy-tear perforated rolls allow fast, convenient customization
- Ideal for securing areas where skin may stretch and where repeated taping is needed



“My 1st priority is critical securement.”



3M™ Durapore™ Silk-like Tape

- Silk-like fabric tape features strong, reliable adhesion
- Durable fabric resists stretching and provides convenient bi-directional tear capabilities
- Conforms well to highly contoured areas
- Ideal for securing tubing and devices, anchoring bulky dressings, immobilizing fingers and toes, and stabilizing finger splints
- Not recommended for extremely moist conditions



3M™ Universal Cloth Adhesive Tape (UCAT)*

- Strong, reliable cloth tape adheres well even in extremely moist conditions
- Durable fabric resists stretching
- Ideal for securing critical tubing and positioning or immobilizing body parts

* This Product Contains Dry Natural Rubber



“I need a tape for a specialty application.”



3M™ Microfoam™ Tape

- Highly conformable elastic foam tape stretches in all directions to accommodate swelling and promote comfort
- Gentle, secure adhesion to irregularly contoured sites
- Ideal for compression applications, providing cushioning or securing dressings where movement is anticipated



3M™ Medipore™ H Soft Cloth Tape

- High strength adhesive ensures reliable securement for both moist and dry skin
- Porous, soft cloth fabric allows skin to breathe, helping maintain skin integrity
- Stretches across and diagonally to accommodate swelling, distension and movement
- Easy-tear perforated rolls allow fast, convenient customization
- Ideal for securing areas where skin may stretch and where repeated taping is needed


How to Use Tape Effectively

1. Dry, clean skin




Tape sticks best on clean, dry skin. Allow skin preps to dry before applying tape.

2. Use firm pressure

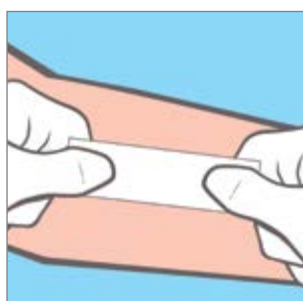

Apply tape to skin without stretching. Rub gently but firmly to stick tight.

3. Easy removal tip

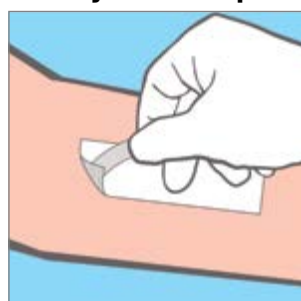

To remove tape that's stuck down hard, try lifting a corner with a small piece of fresh tape.

4. Low and slow



Remove tape by pulling slowly back over itself. Hold skin down at peel edge.

At-Risk Skin	3M™ Kind Removal Silicone Tape	Cat. No.	NHRIC/NDC No.	Size	Rolls/Box	Boxes/Case	HCPCS Code	
	Everyday Use		Standard Rolls:					
2770-1			8333-2770-01	1 in. x 5.5 yd./2,5 cm x 5 m	12	10	A4452	
2770-2			8333-2770-02	2 in. x 5.5 yd./5,0 cm x 5 m	6	10	A4452	
Single Use Rolls:								
2770S-1			8333-2770-11	1 in. x 1.5 yd./2,5 cm x 1,3 m	100	5	A4452	
2770S-2			8333-2770-12	2 in. x 1.5 yd./5,0 cm x 1,3 m	50	5	A4452	
Critical Securement			Standard Rolls:					
			1534-0	8333-1534-00	1/2 in. x 10 yd./1,25 cm x 9,1 m	24	10	A4452
			1534-1	8333-1534-01	1 in. x 10 yd./2,5 cm x 9,1 m	12	10	A4452
			1534-2	8333-1534-02	2 in. x 10 yd./5 cm x 9,1 m	6	10	A4452
			1534-3	8333-1534-03	3 in. x 10 yd./7,6 cm x 9,1 m	4	10	A4452
			Specialty Applications		Standard Rolls:			
	2861	8333-2861-01			1 in. x 10 yd./2,5 cm x 9,1 m	24	1	A4452
	2862	8333-2862-01			2 in. x 10 yd./5,0 cm x 9,1 m	12	1	A4452
	2863	8333-2863-01			3 in. x 10 yd./7,6 cm x 9,1 m	12	1	A4452
	2864	8333-2864-01			4 in. x 10 yd./10,1 cm x 9,1 m	12	1	A4452
	2866	8333-2866-01			6 in. x 10 yd./15,2 cm x 9,1 m	12	1	A4452
	Critical Securement			Standard Rolls:				
1538-0				8333-1538-00	1/2 in. x 10 yd./1,25 cm x 9,1 m	24	10	A4452
1538-1				8333-1538-01	1 in. x 10 yd./2,5 cm x 9,1 m	12	10	A4452
1538-2				8333-1538-02	2 in. x 10 yd./5 cm x 9,1 m	6	10	A4452
1538-3				8333-1538-03	3 in. x 10 yd./7,6 cm x 9,1 m	4	10	A4452
Single Use Rolls:								
Specialty Applications			Standard Rolls:					
			1538S-1	8333-1538-11	1 in. x 1-1/2 yd./2,5 cm x 1,37 m	100	5	A4452
			1538S-2	8333-1538-12	2 in. x 1-1/2 yd./5 cm x 1,37 m	50	5	A4452
			Single Use Rolls:					
			881-1	8333-0881-10	1 in. x 10 yd./2,5 cm x 9,1 m	12	12	A4452
			881-2	8333-0881-20	2 in. x 10 yd./5 cm x 9,1 m	6	12	A4452
	Specialty Applications		Standard Rolls:					
			881-3	8333-0881-30	3 in. x 10 yd./7,5 cm x 9,1 m	4	12	A4452
			<i>*This Product Contains Dry Natural Rubber.</i>					
			Standard Rolls:					
			1528-1	8333-1528-01	1 in. x 5-1/2 yd./2,5 cm x 5 m*	12	6	A4452
			1528-2	8333-1528-02	2 in. x 5-1/2 yd./5 cm x 5 m*	6	6	A4452
Specialty Applications			Standard Rolls:					
			1528-3	8333-1528-03	3 in. x 5-1/2 yd./7,6 cm x 5 m*	4	6	A4452
			1528-4	8333-1528-04	4 in. x 5-1/2 yd./10,1 cm x 5 m*	3	6	A4452
			<i>*Stretched Length</i>					
			Standard Rolls:					
			2861	8333-2861-01	1 in. x 10 yd./2,5 cm x 9,1 m	24	1	A4452
	2862	8333-2862-01	2 in. x 10 yd./5,0 cm x 9,1 m	12	1	A4452		
	2863	8333-2863-01	3 in. x 10 yd./7,6 cm x 9,1 m	12	1	A4452		
	2864	8333-2864-01	4 in. x 10 yd./10,1 cm x 9,1 m	12	1	A4452		
	2866	8333-2866-01	6 in. x 10 yd./15,2 cm x 9,1 m	12	1	A4452		
	2868	8333-2868-01	8 in. x 10 yd./20,3 cm x 9,1 m	6	1	A4452		
	Single Use Rolls:							
2862S	8333-2862-03	2 in. x 2 yd./5,0 cm x 1,8 m	48	1	A4452			
2864S	8333-2864-03	4 in. x 2 yd./10,1 cm x 1,8 m	24	1	A4452			
2866S	8333-2866-03	6 in. x 2 yd./15,2 cm x 1,8 m	16	1	A4452			



3M Critical & Chronic Care Solutions Division
 3M Health Care
 2510 Conway Ave.
 St. Paul, MN 55144
 1-800-228-3957
 www.3M.com/MedicalTapes

3M, Durapore, Medipore, Microfoam and Transpore are trademarks of 3M. Please recycle. Printed in U.S.A.
 © 2012, 2014 3M. All rights reserved.
 70-2010-9103-3

For more detailed product information and proper taping techniques visit:
www.3M.com/MedicalTapes