

# 3M™ Surgical Clipper Classic 9602/9603

For use with 9600 blade



Corded and  
cordless options  
**increase**  
OR flexibility

- Removes dry body hair
- Easy blade attachment and removal
- Clean with alcohol or disinfectant wipes
- Available in corded and cordless models
- Cordless model (9602) provides approximately 30-45 minutes of cordless run time on a 14 hour recharge
- Corded model (9603) provides continuous run time

## Ordering Information

Catalog Number	Product Description	Items/Box	Boxes/Case
9600	3M™ Surgical Clippers Classic Blade 9600	40	1
9602	3M™ Surgical Clipper Classic 9602 (rechargeable with 6 ft. cord for recharging)	1	1
9603	3M™ Surgical Clipper Classic 9603 (cord-operated with 10 ft. cord)	1	1
9611	3M™ Surgical Clipper Classic Wall Bracket 9611	1	1

For more information about the 3M™ Surgical Clipper Classic 9602/9603 or other 3M Health Care products:

- Contact your local 3M sales representative
- Call the 3M Health Care Customer Helpline at 1-800-228-3957
- Visit [3M.com/infectionprevention](http://3M.com/infectionprevention)



# How to use

3M™ Surgical  
Clipper Classic  
9602/9603

For use with  
9600 blade

## Changing Blades



### Attach

Remove backing from the blade packaging, hold clipper body at a 45-degree angle and slide into single-use blade.



### Remove

Use your thumb to slide the used blade into a sharps waste receptacle.

NOTE: Always use a new blade for each patient to avoid cross-contamination.

## Clipping



### Push

Position the clipper like a pencil with the blade at a 15-30 degree angle to the skin. Gently push the clipper forward, against the direction of the patient's hair growth.

NOTE: Do not use excessive pressure to push the clipper down on the patient's skin.

## Cleaning



### Wipe

With the clipper in the "off" position and the blade removed, wipe the clipper handle with an alcohol or disinfectant wipe.

NOTE: Do not hold under running water or submerge clipper.

## Charging



### Charge

Before initial use, charge the 9602 model for 18-24 hours. After each use, store the clipper body in the "off" position and plug the charge cord into an electrical outlet. The 9602 provides approximately 30-45 minutes of cordless run time on a 14 hour recharge. The corded 9603 model provides unlimited, continuous run time and needs to be plugged in only when in use.

NOTE: There is no risk of overcharging, but after initial use, charging the 9602 for more than 14 hours will not result in longer run time.

Refer to package insert for warnings and more detailed instructions.



**3M Health Care**  
**Infection Prevention Division**  
3M Center, Building 275-4E-01  
St. Paul, MN 55144-1000  
U.S.A.  
800-228-3957  
3M.com/infectionprevention

**Medical Division**  
**3M Canada**  
Post Office Box 5757  
London, Ontario N6A 4T1  
Canada  
800-563-2921

3M is a trademark of 3M.  
Please recycle. Printed in U.S.A.  
© 3M 2012. All rights reserved.  
70-2010-9137-1