

# 3M™ Surgical Clipper Professional 9681

For use with 9680 blade



Superior design  
and performance  
**minimize**  
nicks and cuts

- 9680 blade design helps prevent skin from being pulled into the cutting blade
- Away-from-the-skin blade positioning minimizes the potential for nicks or cuts
- In clinical studies, the 3M™ Clipper Professional 9681 demonstrated a nick rate of just 5%, compared with the CareFusion® 4406 General Purpose Blade at 25% and the Medline™ DYND70850 Universal Blade at 12.5%<sup>1</sup>
- Single blade for standard and trauma procedures
- Removes all types of hair in a single pass
- 160 minutes of use time on a 3-hour recharge
- LED indicator lights provide battery charge status

## Ordering Information

Catalog Number	Product Description	Items/Box	Boxes/Case
9680	3M™ Surgical Clipper Professional Blade 9680	50	1
9681	3M™ Surgical Clipper Professional 9681	1	1
9682	3M™ Surgical Clipper Professional Drop-in Charger Stand with Cord for 9681	1	1

For more information about the 3M™ Surgical Clipper Professional 9681 or other 3M Health Care products:

- Contact your local 3M sales representative
- Call the 3M Health Care Customer Helpline at 1-800-228-3957
- Visit [3M.com/infectionprevention](http://3M.com/infectionprevention)

## Reference

<sup>1</sup> Study EM-05-012508



# How to use

3M™ Surgical  
Clipper  
Professional  
9681

For use with  
9680 blade

## Changing Blades



### Attach

Remove backing from the blade packaging. Without touching the blade, hold the clipper body over the blade and slide into position until it clicks.

NOTE: Before attaching or removing blade, be sure clipper button is in OFF position.



### Remove

Hold the blade facing down. Push the blade release button with your thumb, allowing the used blade to fall into a sharps waste receptacle.

WARNING: Do not reuse the clipper blade assembly. Single patient use only.

## Clipping



### Push

Position the clipper like a pencil with the blade flat or at a slight angle on the skin surface. Gently push the clipper forward against the direction of the patient's hair growth.

NOTE: Do not use excessive pressure to push the clipper down on the patient's skin.



### Pull

Position the blade flat or at a slight angle on the skin surface and gently pull the clipper toward you.

NOTE: Do not rake the blade across the patient's skin. Clipper will not cut hair if used incorrectly.

## Cleaning Options



### Rinse

With the clipper turned "off" and the blade removed, rinse the handle under warm running water. Ensure clipper is thoroughly dry before recharging or reusing.

NOTE: If choosing to rinse, rinsing should be completed prior to disinfecting with a wipe or submersion.



### Wipe

With the clipper in the "off" position and the blade removed, wipe the clipper handle with an alcohol or disinfectant wipe. Ensure clipper is thoroughly dry before recharging or reusing.

OR



### Submerge

With the clipper in the "off" position and the blade removed, soak the clipper in a disinfectant solution. Ensure clipper is thoroughly dry before recharging or reusing.

CAUTION: Do not submerge clipper in water or other solution deeper than 6 inches or longer than 15 minutes.

## Charging



### Charge

Before initial use, charge clipper for 3 hours. After each use, store the handle in the "off" position in the plugged-in charge stand. Clipper provides 160 minutes of cordless use time on a 3-hour recharge.

When fully charged, the top three LED light gauges will be green and the bottom two will be yellow. When battery needs recharging, the lowest LED light will blink. During recharge, LED lights will blink. The level of the charge can be determined by the light blinking at that time.

Refer to package insert for warnings and more detailed instructions.

## 3M

**3M Health Care  
Infection Prevention Division**  
3M Center, Building 275-4E-01  
St. Paul, MN 55144-1000  
U.S.A.  
800-228-3957  
3M.com/infectionprevention

**Medical Division  
3M Canada**  
Post Office Box 5757  
London, Ontario N6A 4T1  
Canada  
800-563-2921

3M is a trademark of 3M.  
Please recycle. Printed in U.S.A.  
© 3M 2014. All rights reserved.  
70-2010-9134-8