


3M Infection Prevention Solutions

3M™ Surgical Clipper Professional 9681

For use with 9680 blade

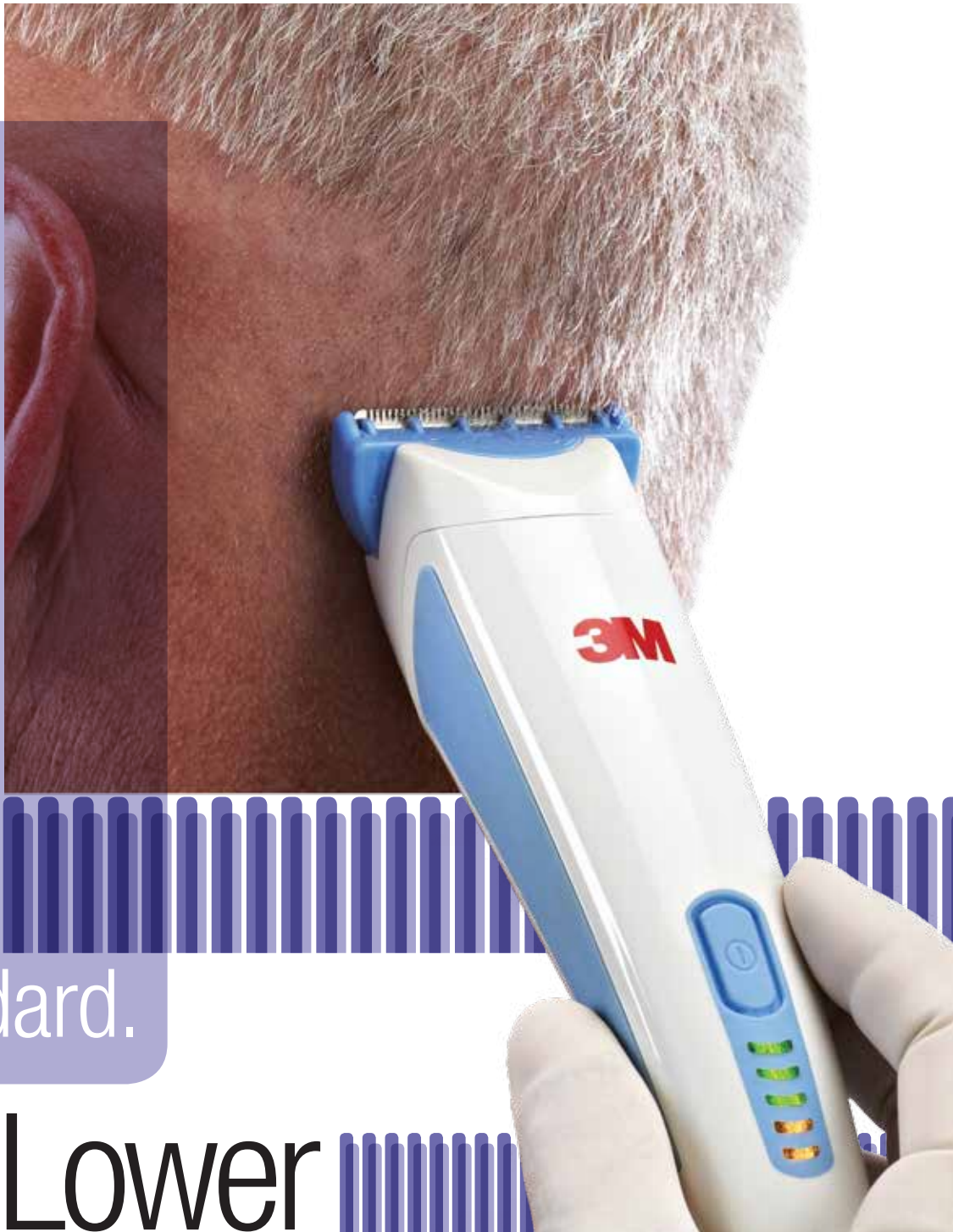

Raise
the
standard.

Lower
your
nick rate.

3M



Clip without compromise.



3M™ Surgical Clipper Professional 9681 combines superior design and performance to help reduce the risk of infection, streamline workflow and promote a positive perioperative experience for OR staff, surgeons and patients.

At-a-glance


- 9680 blade helps prevent skin from being pulled into the cutting blade
- Away-from-the-skin blade positioning minimizes the potential for nicks or cuts
- In clinical studies, the 3M™ Surgical Clipper Professional 9681 demonstrated a nick rate of just 5%, compared with the CareFusion® 4406 General Purpose Blade at 25% and the Medline™ DYND70850 Universal Blade at 12.5%¹
- Single blade for standard and trauma procedures
- Removes all types of hair in a single pass
- 160 minutes of use time on a 3-hour recharge
- LED indicator lights provide battery charge status





Low nick rate

The patent-pending 9680 blade is uniquely designed to prevent skin from being pulled into the cutting blade. Paired with the blade's away-from-the-skin positioning, the 3M™ Surgical Clipper Professional 9681 demonstrated a nick rate of just 5% in clinical studies, compared with the CareFusion® 4406 General Purpose Blade at 25% and the Medline™ DYND70850 Universal Blade at 12.5%.¹

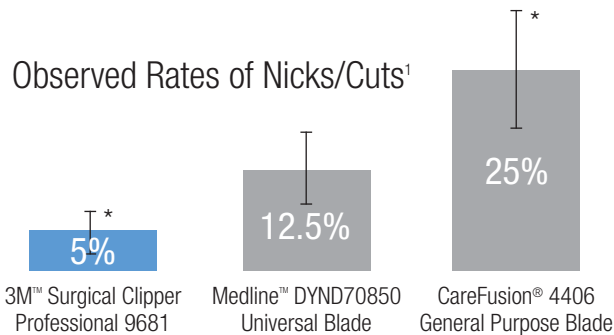

*Difference is statistically significant (P=0.004)



Battery power

Powered by a lithium ion (Li-ion) battery, the 3M™ Surgical Clipper Professional 9681 provides 160 minutes of clipping time on a 3-hour recharge. Unlike nickel-metal hydride (NiMH) batteries, Li-ion batteries do not have a memory effect. This means the clipper always runs at full power even when not fully charged and can be stored in the charge stand without shortening the battery life.



One blade

The 9680 blade is effective for standard clipping and trauma procedures and all types of head and body hair, including sensitive areas of the body. The single-blade approach not only helps the OR standardize clipping protocols, it also eliminates the need to purchase multiple clipper blades.

Surgical Clippers

Features, form and function

Feature	3M™ Surgical Clipper Professional 9681 9680 blade	CareFusion® 4413A 4406 blade	Medline™ DYND70840 DYND70850 blade
Head hair	Yes	No	No
Cutting width	36.4 mm	31.8 mm ²	33.0 mm ²
Blade	One-handed slide on/off	Match alignment marks	Push on/press release
Cutting blade position	Away from skin	Next to skin	Away from skin
Nick rate	5% ¹	25% ¹	12.5% ¹
Speed	356 mm/sec ²	160 mm/sec ²	37 mm/sec ²
Cordless use time	160 minutes	60 minutes	60 minutes
Recharge time	3 hours	12 hours	10 hours
Overcharge protection	Yes	No	No
Cleaning	Rinse/submerge/disinfect	Rinse/submerge/disinfect ³	Rinse/disinfect ⁴
Non-slip grip	Yes	No	No
Battery type	Li-ion	NiMH	NiMH
Warranty	2 year	90 days	1 year

Ordering Information

Catalog Number	Product Description	Items/Box	Boxes/Case
9680	3M™ Surgical Clipper Professional Blade 9680	50	1
9681	3M™ Surgical Clipper Professional 9681	1	1
9682	3M™ Surgical Clipper Professional Drop-in Charger Stand with Cord for 9681	1	1

For more information about the 3M™ Surgical Clipper Professional 9681 or other 3M Health Care products:

- Contact your local 3M sales representative
- Call the 3M Health Care Customer Helpline at 800-228-3957
- Visit 3M.com/infectionprevention

References

- 1 Study EM-05-012508
 2 Verified by 3M testing
 3 CareFusion product information
 4 Medline product information



Infection Prevention Division
3M Health Care
 3M Center, Building 275-4E-01
 St. Paul, MN 55144-1000
 U.S.A.
 800-228-3957
3m.com/infectionprevention

Medical Division
3M Canada
 Post Office Box 5757
 London, Ontario N6A 4T1
 Canada
 800-563-2921

Please recycle. Printed in U.S.A.
 © 3M 2012. All rights reserved.
 70-2010-9139-7