

3M™ Tegaderm™ Superabsorber Dressing

# Tegaderm™ Superabsorber Dressing



## Locks in Exudate

Tegaderm™ Superabsorber Dressing provides confidence that the dressing will absorb and contain exudate — even under compression.

- Versatile for a variety of wounds with moderate and high exudate levels
- Helps prevent maceration of the wound edges
- Reduces frequency of dressing changes
- Provides patient comfort

**3M**

# Innovative Design Absorbs and Contains Exudate — Even Under Compression

## Manage Highly Draining Wounds with Confidence



**Diabetic  
(Neuropathic)  
Foot Ulcer**



**Venous  
Leg Ulcer**



**Dehisced  
Surgical  
Wound**



**Pressure Ulcer  
Stage III**



**Pressure Ulcer  
Stage IV**



**Pressure Ulcer  
Unstageable due  
to necrotic base**

## What does Tegaderm™ Superabsorber Dressing do?


- Absorbs and retains exudate in the dressing minimizing the risk of maceration of the wound edges<sup>1</sup>, even under compression.
- Maintains a moist wound environment.
- Removes exudate away from the skin surface through rapid, vertical wicking.
- Traps the wound fluid and wound fluid components away from the wound<sup>2</sup>.



before it becomes saturated.


1. Clinical experiences of using a super absorbing dressing. Meuleneire F. Poster. EWMA Congress 2010, Geneva, Switzerland.

2. The inhibition of matrix metalloproteinase activity in chronic wounds by a polyacrylate superabsorber. Eming S, Smola H, Hartmann B, Malchau G, Wegner R, Krieg T, Smola-Hess S (2008) Biomaterials Jul 29 (19): 2932-2940

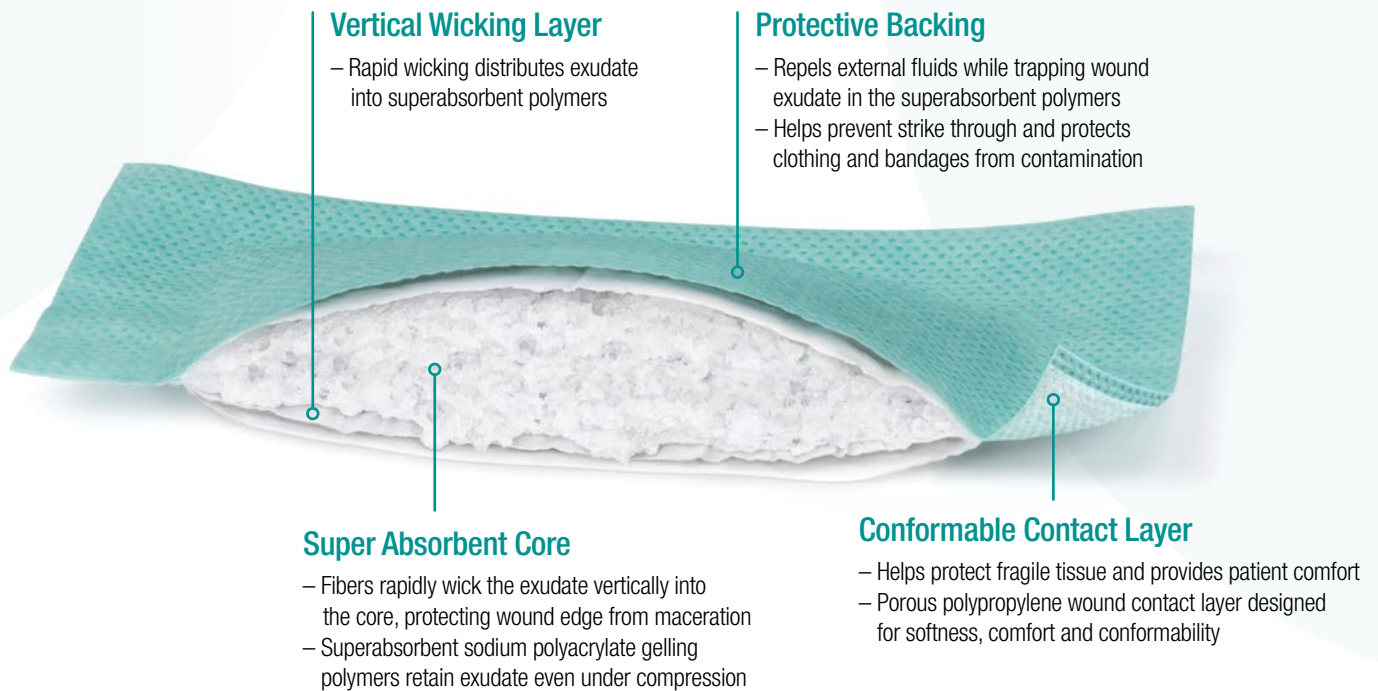


## 3M™ Tegaderm™ Superabsorber Dressing Helps Prevent Maceration

**Mean (SEM) Free Swell Absorbency [g/100 cm<sup>2</sup>]**

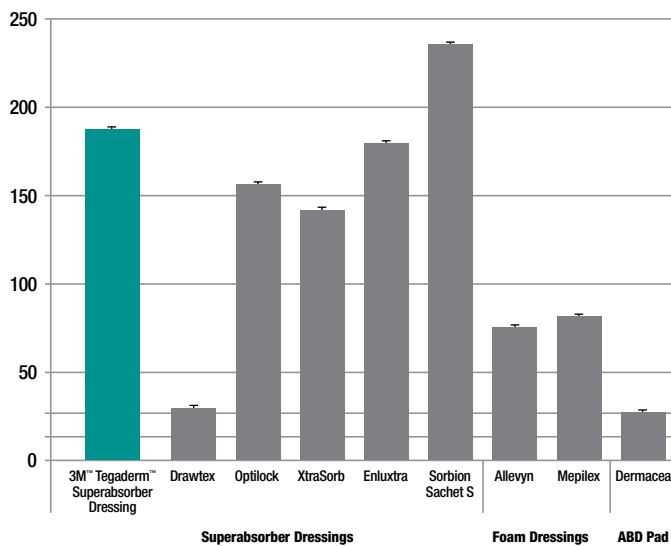

Data on file at 3M.

**Mean (SEM) Fluid Retention Capacity [g/g]**

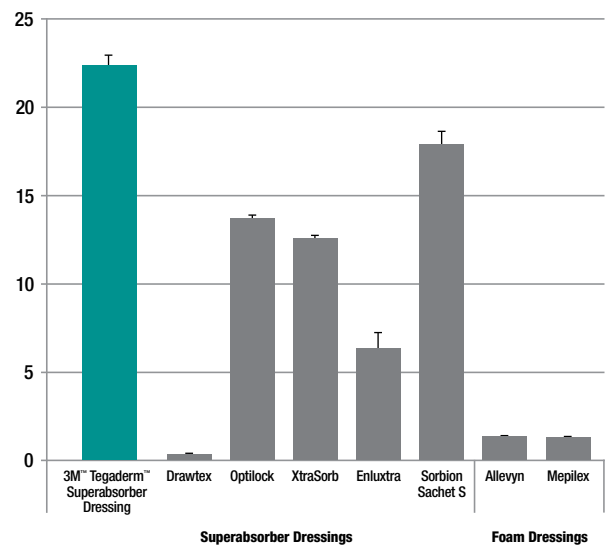

Data on file at 3M.


High absorption and superior exudate retention provides confidence to manage exudate and supports patient comfort with fewer disruptions. The innovative design of Tegaderm™ Superabsorber Dressing helps prevent complications related to periwound maceration.

# Easy and Efficient to Use

## 3M™ Tegaderm™ Superabsorber Dressing

Tegaderm™ Superabsorber dressing can be used both as a primary or secondary dressing. Deep wounds require a wound filler to be applied, since the Superabsorber dressing may not be used as a sponge.

- 1 Select a dressing size that allows 1 inch (2.5 cm) overlap onto intact skin.
- 2 For wounds with depth, use absorbent wound fillers such as 3M™ Tegaderm™ Alginate Dressing or 3M™ Tegaderm™ Alginate Ag Silver Dressing.
- 3 Apply dressing with white contact layer towards the wound — the green hydrophobic backing protects against strike through.
- 4 Secure the dressing by taping the edges with preferred adhesive or other securement, e.g. 3M™ Kind Removal Silicone Tape, 3M™ Tegaderm™ Transparent Film Roll, 3M™ Medipore™ H Soft Cloth Surgical Tape or under 3M™ Coban™ 2 Layer Compression System.

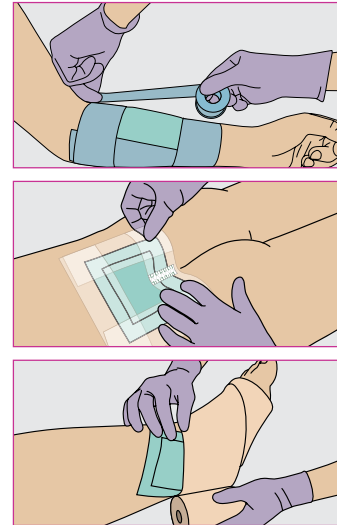

## 3M™ Tegaderm™ Superabsorber Dressing is compatible with other 3M products

- 3M™ Cavilon™ No Sting Barrier Film
- 3M™ Coban™ 2 Layer Compression System
- 3M™ Kind Removal Silicone Tape
- 3M™ Medipore™ H Soft Cloth Surgical Tape
- 3M™ Tegaderm™ Ag Mesh Dressing with Silver
- 3M™ Tegaderm™ Alginate Ag Silver Dressing
- 3M™ Tegaderm™ Alginate Dressing
- 3M™ Tegaderm™ Transparent Film Roll
- 3M™ Wound Cleanser

## Ordering Information

Item Number	Overall Dressing Size	Superabsorber Size	Items/Box	Boxes/Case	HCPCS Code*
90701	3-7/8 in. x 3-7/8 in. (10 cm x 10 cm)	2-3/4 in. x 3-1/8 in. (7 cm x 8 cm)	10	5	A6196
90702	3-7/8 in. x 7-7/8 in. (10 cm x 20 cm)	3-1/8 in. x 6-5/8 in. (8 cm x 17 cm)	10	5	A6197
90703	7-7/8 in. x 7-7/8 in. (20 cm x 20 cm)	6-1/4 in. x 6-5/8 in. (16 cm x 17 cm)	10	5	A6197
90704	7-7/8 in. x 11-3/4 in. (20 cm x 30 cm)	6-1/4 in. x 10-5/8 in. (16 cm x 27 cm)	10	5	A6198

\*Disclaimer: The reimbursement information provided by 3M Health Care and its representatives is intended to provide general information relevant to coding for 3M products. Insurers' reimbursement policies can vary, however, and the use of the codes discussed here does not guarantee that an insurer will cover or pay at any particular level.

For more information, contact your 3M Health Care representative, call the 3M Health Care Customer Helpline at 1-800-228-3957 or visit us at [www.3M.com/C3SD](http://www.3M.com/C3SD).



**Critical & Chronic Care Solutions Division**  
**3M Health Care**  
 2510 Conway Avenue  
 St. Paul, MN 55144  
 USA  
 1-800-228-3957  
[www.3M.com/C3SD](http://www.3M.com/C3SD)

**3M™ Tegaderm™** SIMPLE. DEPENDABLE. TRUSTED.  
 Dressings

3M, Cavilon, Coban, Medipore and Tegaderm are trademarks of 3M. Allewyn is a trademark of Smith & Nephew, Dermacea is a trademark of Covidien, Drawtex is a trademark of Beier Drawtex Healthcare, Enluxtra is a registered trademark of BASF, Mextra and Mepilex are trademarks of Mölnlycke, Optilock is a trademark of Medline, Xtrasorb is a trademark of Derma Sciences. Please recycle. Printed in U.S.A.  
 © 3M 2013, 2014. All rights reserved. 70-2010-9302-1