

Clinpro™ Prophy Paste

From

Coarse to

fine



Stain removing power
of a **coarse paste**
with less abrasion

Give your Patients the best of both worlds

Why choose between a coarse or fine grit Prophy Paste when you can use both?

The unique perlite particles within Clinpro™ Prophy Paste are formulated to break down as you polish from a coarse to fine grit. This unique feature provides the stain removal of a coarse paste with less abrasion to the enamel and dentin.

From
Coarse to *fine*



Clinpro Prophy Paste unique perlite particles start out coarse and break down to a fine paste as you clean and polish.

A Prophy Paste that meets all your needs

It's more than revolutionary.

It also incorporates the same features you'd expect from any prophy paste.

Using Clinpro Prophy Paste results in less gloss reduction which means the high aesthetics and luster of composite restorations will continue to enhance your patient's smile.

- » Contains fluoride (1.23%)
- » Low splatter
- » A Neutral pH value
- » Refreshing mint and bubblegum flavours
- » Available in unit dose cups



Flavours

Bubblegum



Mint




Patented Technology


We call it integrated abrasion variability. It's a mouthful. But it means that you can offer your patients better care. You'll notice the difference the first time you use it.

The unique perlite particles within the paste start out coarse and gradually break down into smaller particles as you clean, resulting in less abrasion to the dentin and enamel. Your patients get a great cleaning and a gentle polishing from the same paste.

Pre Use

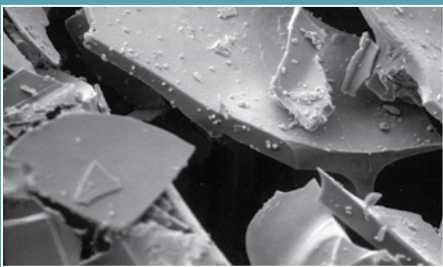

After 15 Seconds

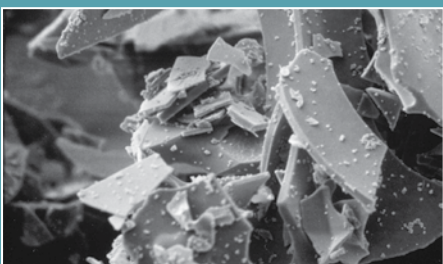

Large cleaning particles within the paste (magnified 3000x) break into fine polishing particles in a matter of seconds.


The benefits show.

Compare Clinpro™ Prophy Paste to the leading pastes.

3M ESPE Clinpro™ Prophy Paste removed stains as well as NUPRO® Coarse, while being significantly less abrasive to both dentin and enamel.

STAIN REMOVAL


Computer Image Analysis Technique. University of Minnesota School of Dentistry MDRCBB.

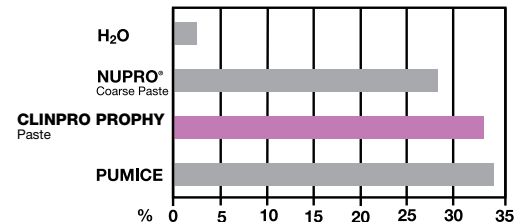

LOSS OF GLOSS REFLECTANCE

Laser Reflectometry on 3M ESPE's Silux Plus™ Anterior Restorative.


Study by University of Minnesota School of Dentistry MDRCBB.

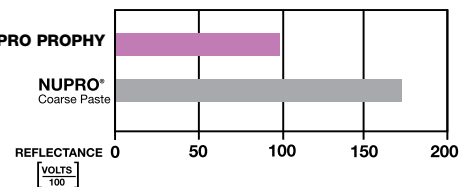

RELATIVE DENTIN ABRASION

Radiotracer Method In Accordance to ADA Specifications.


Indiana University Oral Health Research Institute.

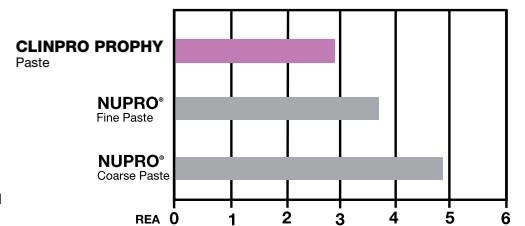

RELATIVE ENAMEL ABRASION

Radiotracer Method In Accordance to ADA Specifications.

Indiana University Oral Health Research Institute.

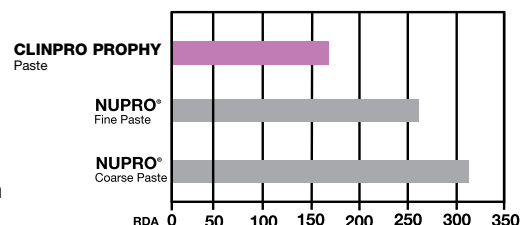

“Clinpro Propphy Paste is an outstanding alternative for cleaning and polishing both natural teeth, direct, and indirect restorations.”

Trish Jones, RDH, BS, Technical Advisor for Aurum Ceramic @ LVI

Ordering Information



Item Number

Product Information

12611

Clinpro™ Propphy Paste, mint flavour
Includes 200 unit dose cups and one propphy ring.

12612

Clinpro™ Propphy Paste, bubblegum flavour
Includes 200 unit dose cups and one propphy ring.

For a FREE sample pack of Clinpro Propphy Paste
(1 cup mint flavour paste, 1 cup bubblegum flavour paste)
Contact your Product Specialist

3M ESPE

3M Australia Pty Limited.
Bldg A
1 Rivett Road
North Ryde NSW 2113
ABN: 90 000 100 096
Ph 1300 363 454

3M and Clinpro are trademarks of 3M.
Please recycle. Printed in Australia.
© 3M 2011. All rights reserved.
PB5218/0711