

3M Science.
Applied to Life.™



3M™ Pentamix™ Lite Automatic Mixing Unit


Start automixing and simplify your life.

With the touch of a button, you can mix and dispense impression material. Doctors get more usable working time while assistants have less stress and hand fatigue. In the end, your material will be homogeneous and void-free which means the finals will fit better.


3M makes it easy to start automixing with our affordable Pentamix Lite mixing unit. It works with all 3M materials including 3M™ Imprint™ 4 VPS and 3M™ Impregum™ Polyether Impression Materials. Plus, with its convenient hand grip, the device can be easily carried from operator to another. Just plug in, switch on and start mixing!




Fast and intuitive set-up AND USE

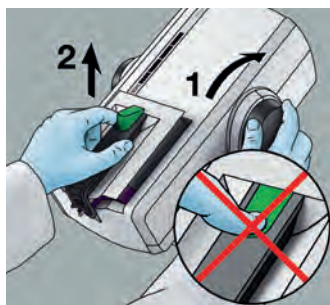


1. Remove the cartridge.

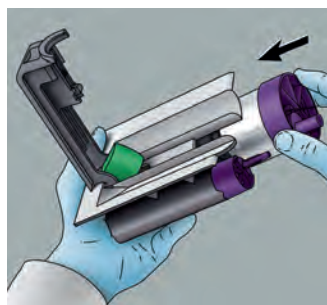


2. Insert 3M™ Penta™ Foil Bag with the 3M Impression Material of your choice into the cartridge.

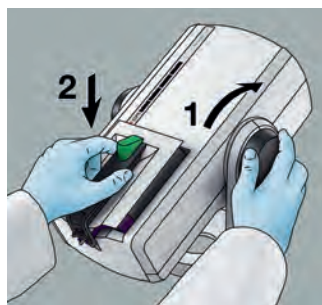

3. Insert the cartridge into the 3M™ Pentamix™ Lite Automatic Mixing Unit.

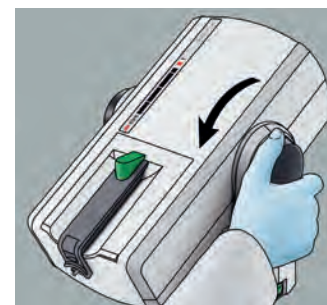


4. Turn the wheel (piston) forward.

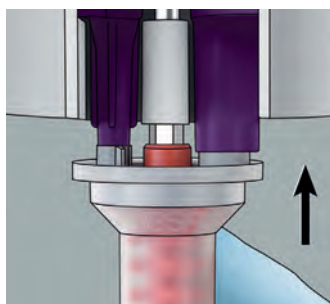

5. Attach a 3M™ Penta™ Mixing Tip – Red.



6. Close the latch with an audible click.

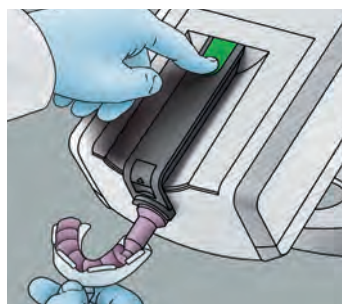

7. Press the green start button to fill the impression tray or elastomer syringe.

Ordering information

Item #	Product Information
77907	3M™ Pentamix™ Lite Automatic Mixing Unit – 120 V/60 Hz Includes: 1 3M™ Penta™ Cartridge for Pentamix Lite
77907	13M™ Penta™ Cartridge for 3M™ Pentamix™ Lite
77949	3M™ Penta™ Mixing Tips – Red Includes: 50 Mixing Tips
3M™ Impregum™ Penta™ Soft Polyether Impression Material Refills Each refill includes: 2 base pastes – 300 ml each, 2 catalysts – 60 ml	
31741	3M™ Impregum™ Penta™ Soft Heavy Body Refill
31769	3M™ Impregum™ Penta™ Soft Quick Step Heavy Body Refill
31731	3M™ Impregum™ Penta™ Soft Medium Body Refill
31765	3M™ Impregum™ Penta™ Soft Quick Step Medium Body Refill
31642	3M™ Impregum™ Penta™ Medium Body Refill
3M™ Paradigm™ Penta™ VPS Impression Refills Each refill includes: 2 – 300ml base pastes; 2 – 60ml catalysts	
36716	3M™ Paradigm™ Penta™ Heavy Body
36717	3M™ Paradigm™ Penta™ Heavy Body Fast Set

Item #	Product Information
3M™ Imprint™ 4 Penta™ VPS Impression Refills—Tray Materials Each refill includes: 2 – 300ml base pastes; 2 – 60ml catalysts	
71484	3M™ Imprint™ 4 Penta™ Heavy
71485	3M™ Imprint™ 4 Penta™ Super Quick Heavy
71486	3M™ Imprint™ 4 Penta™ Putty
3M™ Imprint™ 4 VPS Impression Refills—Wash Materials Each refill includes: 4 – 50ml cartridges; 5 – mixing tips, yellow; 5 – 3M™ Intra-Oral Syringes, green	
71488	3M™ Imprint™ 4 Light
71487	3M™ Imprint™ 4 Regular
71490	3M™ Imprint™ 4 Super Quick Light
71489	3M™ Imprint™ 4 Super Quick Regular
3M™ Imprint™ 4 Preliminary Penta™ VPS Impression Refills Each refill includes: 2 – 300ml base pastes; 2 – 60ml catalysts	
71521	3M™ Imprint™ 4 Preliminary Penta™
71522	3M™ Imprint™ 4 Preliminary Penta™ Super Quick
3M™ Imprint™ 4 Preliminary Penta™ VPS Impression Refills—Value Packs Each refill includes: 6 – 300ml base pastes; 6 – 60ml catalysts	
71523	3M™ Imprint™ 4 Preliminary Penta™
71524	3M™ Imprint™ 4 Preliminary Penta™ Super Quick



3M Oral Care
2510 Conway Avenue
St. Paul, MN 55144-1000 USA

Phone 1-800-634-2249
Web 3M.com/dental

3M Canada
Post Office Box 5757
London, Ontario N6A 4T1
Canada

Phone 1-888-363-3685

Customer Care Center: 1-800-634-2249
3M.com/greatimpressions

3M, Impregum, Imprint, Penta and Pentamix are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. Please recycle. Printed in U.S.A. © 3M 2016. All rights reserved.
70-2013-0603-5 (Rev. 8/2016)