

# Pentamix™ Lite

Automatic Mixing Unit



# START AUTOMIXING

and simplify your life.

**3M** ESPE

# 3M ESPE's ENTRY MODEL to the world of AUTOMATIC MIXING

## ONE mixing device FOR ALL 3M ESPE impression materials

Combine the portable power of the new Pentamix™ Lite Automatic Mixing Unit with excellent Imprint™ 4 VPS and Impregum™ Polyether Impression Materials and you get the perfect mix of precision and accuracy. Suitable for all viscosities — from highly viscous putty to medium-bodied materials.

### Imprint™ 4 VPS Impression Material

Designed for **Speed, Control** and **Precision**.

**SPEED.** Imprint 4 material offers the fastest intra-oral setting time among leading VPS fast-setting impression materials.

**CONTROL.** Imprint 4 materials put you in control with adequate working time — a factor for avoiding problems, and for making good impressions with reliable results.

**PRECISION.** Imprint 4 material is super-hydrophilic, and displaces moisture right from the start in the unset state, which helps prevent detail-disrupting voids.

GET FULL CONTROL OF YOUR PROCEDURE WITH PENTAMIX™ LITE ...





With its unique swiveling design, the Pentamix™ Lite Automatic Mixing Unit requires a minimum of space in the upright standby position and it can be stowed in a space-saving way.

Thanks to its low weight and its ergonomic hand grip, the device can also easily be carried from one operator to another — if desired. Just plug in, switch on and start mixing!

**GOES PERFECTLY  
TOGETHER!**

THE PENTAMIX™ LITE SYSTEM ADDS  
• AFFORDABILITY • CONTROL  
• CONVENIENCE  
TO YOUR IMPRESSION WORK



## Impregum™ Polyether Impression Material

With unrivalled product features such as **OUTSTANDING INITIAL HYDROPHILICITY**, **EXCELLENT FLOW** for highest detail reproduction and precision, and **UNIQUE SNAP-SET** behavior, Impregum polyether material helps to achieve high-quality, finely detailed impressions and perfectly fitting restorations, including fewer costly and time-consuming adjustments or reworks.

... AND 3M ESPE IMPRESSION MATERIALS.


# Fast and intuitive set-up **AND USE**



1. Remove the cartridge.




2. Insert Penta™ Foil Bag with the 3M ESPE impression material of your choice into the cartridge.

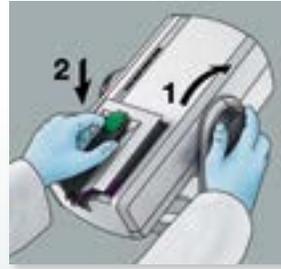

3. Insert the cartridge into the Pentamix™ Lite Automatic Mixing Unit.



4. Turn the wheel (piston) forward.

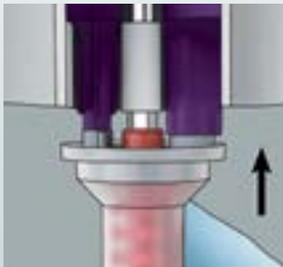

5. Attach a Penta™ Mixing Tip – Red.



6. Close the latch with an audible click.



7. Press the green start button to fill the impression tray or elastomer syringe.

## Penta™ Mixing Tip – Red

for all Pentamix™ Devices

- Enhanced geometry to facilitate insertion in all Pentamix devices
- Grey cover plate for increased insertion visibility
- New authentication label on packaging — ensures that you are using the original



## Ordering Information

### PENTAMIX™ LITE AUTOMATIC MIXING UNIT AND ACCESSORIES

Item No.	Product Information
77907	<b>Pentamix™ Lite Automatic Mixing Unit – 120 V/60 Hz</b> Includes: 1 Penta™ Cartridge for Pentamix™ Lite
77944	<b>1 Penta™ Cartridge for Pentamix™ Lite</b>
77949	<b>Penta™ Mixing Tips – Red</b> Includes: 50 Mixing Tips

### IMPREGUM™ SOFT POLYETHER IMPRESSION MATERIAL

Refills: Each contains: 2 – 300ml base pastes, 2 – 60ml catalysts

Item No.	Product Information
31731	<b>Impregum™ Penta™ Soft Medium Body</b>
31741	<b>Impregum™ Penta™ Soft Heavy Body</b>
31765	<b>Impregum™ Penta™ Soft Quick Step Medium Body</b>
31769	<b>Impregum™ Penta™ Soft Quick Step Heavy Body</b>
31746	<b>Impregum™ Penta™ Soft Light Body</b> Contains: 1 – 300ml base pastes, 1 – 60ml catalysts
31785	<b>Impregum™ Soft Tray Material</b> Contains: 4 – 50ml cartridges, 10 – polyether tray mixing tips, purple

### IMPRINT™ 4 VPS IMPRESSION MATERIAL

Penta™ Refills — Tray Material: Each contains: 2 – 300ml base pastes, 2 – 60ml catalysts

Item No.	Product Information
71484	<b>Imprint™ 4 Penta™ Heavy</b>
71485	<b>Imprint™ 4 Penta™ Super Quick Heavy</b>
71486	<b>Imprint™ 4 Penta™ Putty</b>

Refills — Wash & Tray Materials – Cartridge: Each contains: 4 – 50ml cartridges, 5 – yellow mixing tips, 5 – green 3M™ ESPE™ Intra-Oral Syringes

Item No.	Product Information
71487	<b>Imprint™ 4 Regular</b>
71488	<b>Imprint™ 4 Light</b>
71489	<b>Imprint™ 4 Super Quick Regular</b>
71490	<b>Imprint™ 4 Super Quick Light</b>

### POSITION™ PENTA™ QUICK VPS IMPRESSION MATERIAL

Refill: Contains: 2 – 300ml base pastes, 2 – 60ml catalysts

Item No.	Product Information
29063	<b>Position™ Penta™ Quick VPS Alginate Replacement – Refill</b>

**3M ESPE**

#### Dental Products

2510 Conway Avenue  
St. Paul, MN 55144-1000 USA  
1-800-634-2249

#### 3M Canada

Post Office Box 5757  
London, Ontario N6A 4T1  
Canada  
1-888-363-3685

**Customer Care Center: 1-800-634-2249**

**[www.3MESPE.com/PentamixLite](http://www.3MESPE.com/PentamixLite)**

3M, ESPE, Impregum, Imprint, Penta and Pentamix are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. © 3M 2013. All rights reserved. Please recycle. Printed in U.S.A. 70-2013-0604-3