

Clinpro™ Sealant



So simple you'll be
tickled pink

Summary of advantages

- Colour-change technology to visualise placement
- Convenient direct-delivery syringe for easy application
- Low viscosity to flow easily into pits and fissures
- Contains and releases fluoride
- Bonds to enamel

Protect and defend teeth against decay

With its unique patented colour-change technology 3M™ ESPE™ Clinpro™ Sealant makes sealant application easier than ever. This fluoride-releasing sealant goes on pink and cures to a natural white. Applied with our convenient ultra-fine syringe tip, its low-viscosity formula flows easily into pits and fissures, yet is viscous enough to stay in place during application. Clinpro Sealant requires fewer occlusal adjustments than a filled sealant and provides powerful, long-lasting protection against caries.*

Backed by 10 years of clinical success, 3M ESPE is committed to providing the most advanced and patient-friendly solutions to dental professionals and their patients.

Before



After



Photography by Dr. Chris Bryant

* The American Dental Association (ADA) and Caries Management By Risk Assessment (CAMBRA) recommend sealants as an aid to prevent caries. (3M ESPE Data on file)

Easy-to-use technique

3M™ ESPE™ Clinpro™ Sealant offers colour-change technology for ultimate placement control.

Tooth preparation: Thoroughly clean teeth to remove plaque and debris from enamel surfaces and fissures. Rinse thoroughly with water and dry.

Note: Do not use any cleaning medium that contains oils or fluoride. Prepare adhesive according to dispensing instructions.



1 Apply etchant to all enamel surfaces to be sealed.



2 Thoroughly rinse teeth to remove etchant, then dry the etched surfaces.



3 Apply Clinpro sealant into pits and fissures.



4 Light-cure the sealant with an LED light for the amount of time specified to cure the sealant.

Ordering Information



Item # Product Information

3M™ ESPE™ Clinpro™ Sealant

12646 Introductory Kit, Syringes

Contains: 2 – 1.2 ml Clinpro Sealant Syringes; 1 – 3 ml 3M™ ESPE™ Scotchbond™ Etchant Syringe; 20 – Sealant Syringe Tips (Black); 25 – Etchant Syringe Tips (Blue)

12647 Refill, Syringes

Contains: 1 – 1.2 ml Clinpro Sealant Syringe; 10 – Sealant Syringe Tips (Black)

12641 Introductory Kit, Bottles

Contains: 2 – 6 ml Clinpro Sealant Bottles; 1 – 9 ml 3M™ ESPE™ Scotchbond™ Etchant Bottle; 2 – Brush Handles; 60 – Brush Tips; 50 – Fiber Tips; 1 – Mixing Well with Cover

12642 Refill, Bottle

Contains: 1 – 6 ml Clinpro Sealant Bottle



Get the power of pink!

Contact your local 3M ESPE sales representative to learn more about our full line of preventive products.

3M ESPE

3M Australia Pty Limited

ABN 90 000 100 096
Building A, 1 Rivett Road
North Ryde NSW 2113
Ph: 1300 363 454
www.3MESPE.com.au

3M New Zealand Limited

94 Apollo Drive
Rosedale
Auckland 0632
Ph: 0800 80 81 82
www.3MESPE.co.nz

3M, ESPE, Clinpro, and Scotchbond are trademarks of 3M or 3M ESPE.
Please recycle. Printed in Australia. © 3M 2014.
All rights reserved. PB6757/0514