Imprint[™] 4 Bite

VPS Bite Registration Material



Highly accurate bite registrations

Imprint[™] 4 Bite VPS Material offers you:

- Good flow behavior
 For high accuracy
- Fast intra-oral setting For a faster procedure
- Easy to cut, trim and position
 Due to material hardness and low flexibility
- ► Now scannable For digital access to the digital workflow
- Fresh Material colour For good readibility
- Cartridge delivery
 For simple and convenient mixing and dispensing

Developed for highly accurate bite registrations.

With its great flow properties and no-slump consistency, Imprint 4 Bite material does not lock into undercuts or interdental areas. Material hardness and low flexibility after setting allow for easy cutting and trimming as well as accurate positioning of stone models.

Saves time with a fast intra-oral setting.

Imprint 4 Bite offers a pleasant working time and a short setting time of only 60 seconds, keeping procedures moving quickly while enhancing patient comfort. And, it helps to reduce inaccuracies due to jaw movements during setting.

	Mixing and application	Setting time from start of mixing*
Imprint [™] 4 Bite VPS Bite Registration Material	0:20	1:00

*The stated times are valid at 23°C/74°F. Times depend on ambient temperature.



Indications:

Universal VPS bite registration material – ideally suited for occlusal recording including optical registration of occlusal data for CAD/CAM systems.



Scan of bite registration made with Imprint™ 4 Bite VPS Material.

Get full control of your impression work.

3M ESPE impression solutions are designed to add speed, convenience and precision to the entire procedure.



Imprint[™] 4 Bite VPS Material is part of the successful family of Imprint 4 VPS impression materials. Try also the versatile range of Imprint[™] 4 VPS materials for your everyday precision impressions as well as Imprint 4 Preliminary VPS Material.

Ordering Information



em No.	Product Information	
529	Imprint™ 4 Bite Refill	
	2 cartridges – 50ml each, 5 Garant [™] Mixing Tips green,	
	5 Garant [™] Intra-oral Tips Bite	
460	50 Garant [™] Intra-oral Tips Bite	



3M ESPE

3M Australia Pty Limited ABN 90 000 100 096 Building A, 1 Rivett Rd North Ryde NSW 2113 Ph: 1300 363 454 www.3MESPE.com.au

3M ESPE

lte 71!

714

3M New Zealand Limited 94 Apollo Drive Rosedale Auckland 0632 Ph: 0800 80 81 82 www.3MESPE.co.nz

3M, ESPE, Garant and Imprint are trademarks of 3M or 3M ESPE. © 2014, 3M. All rights reserved. PB6815.05.14