

3M™ Cavilon™ No Sting Barrier Film Application Guide for I.V. Site Protection




3M™ Cavilon™ No Sting Barrier Film 1 mL wand applicator is recommended for skin protection around intravenous catheter sites.

NOTE! The 28 mL spray bottle should not be used for IV site protection.

Application at time of catheter insertion:

1. **Prepare for and follow catheter insertion** procedure per facility protocol.
2. **Apply antiseptic prep** and allow to dry completely. ⌚
3. **Insert catheter.**
4. **Apply barrier film:** Avoiding the area immediately surrounding insertion site (approximately 2cm), apply an even coating of Cavilon No Sting Barrier Film to the area that will be covered by the adhesive dressing/device. If using Tegaderm™ CHG Chlorhexidine Gluconate I.V. Securement Dressing do not apply the film to skin where the CHG gel pad is placed.
5. **Allow Cavilon No Sting Barrier Film to dry completely** (approximately 30 seconds). ⌚
6. **Apply dressing:** Tegaderm™ I.V. Advanced Securement Dressing or Tegaderm™ CHG Chlorhexidine Gluconate I.V. Securement Dressing or 3M™ PICC/CVC Securement System per protocol.

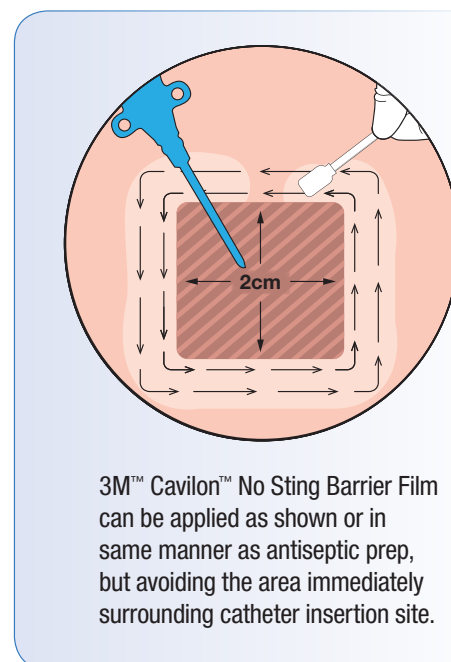

3M™ Cavilon™ No Sting Barrier Film can be applied as shown or in same manner as antiseptic prep, but avoiding the area immediately surrounding catheter insertion site.

Application at dressing change:

1. **Prepare for and follow dressing change procedure** per facility protocol.
2. **Remove securement dressing/device.**
NOTE! Cavilon No Sting Barrier Film forms a protective interface between skin and the adhesive. It is removed with the adhesive product and must be reapplied with each dressing change.
3. **Apply antiseptic prep** and allow to dry completely. ⌚
4. **Apply barrier film:** Avoiding the area immediately surrounding insertion site (approximately 2cm), apply an even coating of Cavilon No Sting Barrier Film to the area that will be covered by the adhesive dressing/device. If using Tegaderm™ CHG Chlorhexidine Gluconate I.V. Securement Dressing do not apply the film to skin where the CHG gel pad is placed.
5. **Allow Cavilon No Sting Barrier Film to dry completely** (approximately 30 seconds). ⌚
6. **Apply dressing:** Tegaderm™ I.V. Advanced Securement Dressing or Tegaderm™ CHG Chlorhexidine Gluconate I.V. Securement Dressing or 3M™ PICC/CVC Securement System per protocol.

3M

Critical & Chronic Care Solutions Division
3M Health Care
2510 Conway Avenue
St. Paul, MN 55144
USA
1-800-228-3957
www.3M.com/Cavilon

3M Canada
P.O. Box 5757
London, Ontario N6A 4T1
Canada
1-800-364-3577
www.3m.ca/skinwound

3M and Cavilon are trademarks of 3M. Used under license in Canada.
Please recycle. Printed in USA. © 3M 2014. All rights reserved.
70-2011-5546-5

⌚ = allow dry time