


Filtek™ Bulk Fill Posterior Restorative




Class I Restoration



1

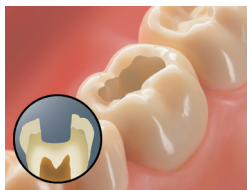

2

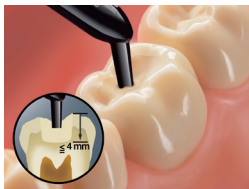

3



4



5



6



7



8


Filtek™ Bulk Fill Posterior Restorative




Class II Restoration



1

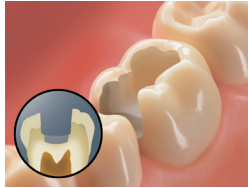


2

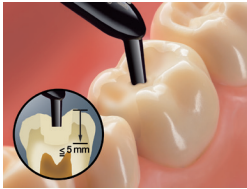

3



4

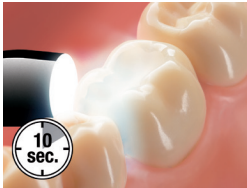

5




6

Filtek™ Bulk Fill Posterior Restorative



Class II Restoration

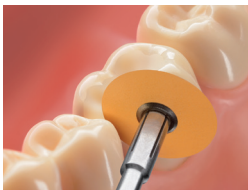

7



8



9



10



11

Filtek™ Bulk Fill Posterior Restorative



Caries Classification	Increment Depth	All halogen lights (with output of 550-1000 mW/cm ²)	3M™ ESPE™ LED lights (with output 1000-2000 mW/cm ²)
Class I	4 mm	40 sec.	20 sec.
Class II	5 mm	20 sec. occlusal, 20 sec. buccal, 20 sec. lingual	10 sec. occlusal, 10 sec. buccal, 10 sec. lingual

3M, ESPE, Adper, Scotchbond and Soft-Lex are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. © 3M 2014. All rights reserved.



3M ESPE
Dental Products
2510 Conway Avenue
St. Paul, MN 55144-1000 USA



3M Deutschland GmbH
Dental Products
Carl-Schurz-Strasse 1
41453 Neuss, Germany

CE
0123

