


## Filtek™ Bulk Fill Flowable Restorative




1



2

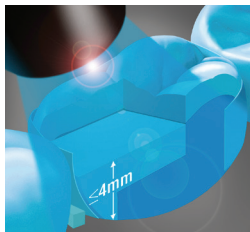

3

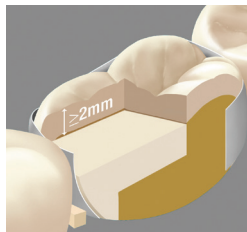


4

Shades	Increment	Cure Time	
		All halogen lights LED lights (with output 550-1000 mW/cm <sup>2</sup> )	Elipar™ S10 (and other 3M ESPE LED lights with output 1000-2000 mW/cm <sup>2</sup> )
U	4mm	20 seconds	10 seconds
A1, A2, A3	4mm	40 seconds	20 seconds


## Filtek™ Bulk Fill Flowable Restorative




1

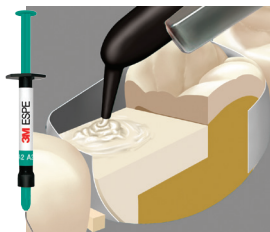

2

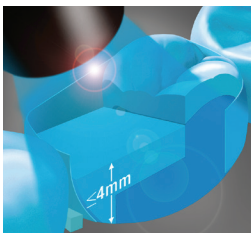

3

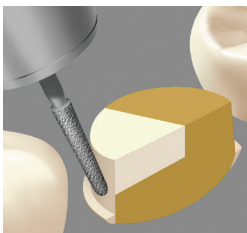

4

Shades	Increment	Cure Time
		All halogen lights LED lights (with output 550-1000 mW/cm <sup>2</sup> )
		Elipar™ S10 (and other 3M ESPE LED lights with output 1000-2000 mW/cm <sup>2</sup> )
U	4mm	20 seconds
A1, A2, A3	4mm	40 seconds

3M, ESPE, Elipar and Filtek are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. © 3M 2014. All rights reserved.

**3M ESPE**

Dental Products  
2510 Conway Avenue  
St. Paul, MN 55144-1000, USA

**3M Deutschland GmbH**

Dental Products  
Carl-Schurz-Strasse 1  
41453 Neuss, Germany

**CE**  
0123

