**Fitting Instructions**  
Must be followed each time respirator is worn.

- Remove respirator from packaging and hold with straps facing upward (1a). Place bottom strap under center flaps next to **WARNING** statement as shown (1b).
- Fully open top and bottom panels, bending nosepiece around thumb at center of foam. Straps should separate when panels are opened. Make certain bottom panel is unfolded and completely opened.
- Place respirator on your face so that the foam rests on your nose and the bottom panel is open under chin. Hold the bottom panel securely under your chin. Pull the top strap over your head and position it high on the back of the head (3a). Then pull the bottom strap over your head and position it around the neck and below the ears (3b). Adjust for a comfortable fit by pulling top panel toward the bridge of the nose and bottom panel under chin.
- Place your fingertips on both hands at the top of the metal nosepiece. Using two hands, mold the nose area to the shape of your nose by pushing inward while moving your fingertips down both sides of the nosepiece.
  - ▲ Pinching the nosepiece using one hand may result in improper fit and less effective respirator performance. Use two hands.
- Perform a User Seal Check prior to each wearing. To check the respirator-to-face seal, cover the middle panel with one or both hands. For the 9210 respirator, inhale and exhale sharply. For the 9211 respirator, inhale sharply. Be careful not to disturb the position of the respirator. If air leaks around the nose, readjust the nosepiece as described in step 4. If air leaks around respirator edges, adjust panels and straps. **If you CANNOT achieve a proper fit, DO NOT enter the contaminated area. See your supervisor.**

**Removal Instructions**

Cup respirator in hand to maintain position on face and pull bottom strap over head. Still holding the respirator in position, pull top strap over head and remove respirator.

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**MIOSH Approved: N95**

At least 95% filtration efficiency against solid and liquid aerosols that do not contain oil.

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3M  
St. Paul, Minnesota, USA  
1-800-243-4630  
9210+ Respirator

THIS RESPIRATOR IS APPROVED ONLY IN THE FOLLOWING CONFIGURATION:

RESPIRATOR COMPONENTS			CAUTIONS AND LIMITATIONS <sup>2</sup>
TC	PROTECTION <sup>1</sup>	FILTERING FACEPIECE	
84A-5688	N95	9210+ X	ABCJMNOP

- PROTECTION**  
N95 - Particulate Filter (95% filter efficiency level) effective against particulate aerosols free of oil; time use restrictions may apply.
- CAUTIONS AND LIMITATIONS**  
A - Not for use in atmospheres containing less than 19.5 percent oxygen.  
B - Not for use in atmospheres immediately dangerous to life or health.  
C - Do not exceed maximum use concentrations established by regulatory standards.  
J - Failure to properly use and maintain this product could result in injury or death.  
M - All approved respirators shall be selected, fitted, used and maintained in accordance with MSHA, OSHA and other applicable regulations.  
N - never substitute, modify, add, or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.  
O - Refer to User's Instructions, and/or maintenance manuals for information on use and maintenance of these respirators.  
P - NIOSH does not evaluate respirators for use as surgical masks.

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