

Choose Filtek™ Bulk Fill Flowable Restorative for fast and simple posterior restorations.



1 Prepare the tooth and apply adhesive.



2 Dispense up to 4 mm of Filtek™ Bulk Fill Flowable Restorative into the prepared cavity and cure.



3 Place a 2 mm (minimum) capping layer of an esthetic composite, such as a Filtek™ universal restorative, and cure.

Clinical photos courtesy of Dr. Reich, Seefeld, Germany

Ordering Information



Filtek™ Bulk Fill Flowable Restorative Refill—Syringe



Filtek™ Bulk Fill Flowable Restorative Refill—Capsule

Filtek™ Bulk Fill Flowable Restorative

Refill	Item #	Item #
SHADE	SYRINGE	CAPSULE
A1	4862A1	4861A1
A2	4862A2	4861A2
A3	4862A3	4861A3
Universal	4862U	4861U
Item #	Product Description	
3740T	Dispensing Tips 20 – 19-gauge dispensing tips	

SYRINGE SHADE REFILLS include 2 – 2g syringes; 20 dispensing tips; Technique Guide; Instructions for Use.

CAPSULE SHADE REFILLS include 15 – 0.2g capsules; Technique Guide; Instructions for Use.

Customer Care Center: 1-800-634-2249 www.3MESPE.com/FiltekBulkFill

★★★★★ Review these products at www.3MESPE.com/review



Dental Products

2510 Conway Avenue
St. Paul, MN 55144-1000 USA
1-800-634-2249

3M Canada


Post Office Box 5757
London, Ontario N6A 4T1
Canada
1-888-363-3685

Please recycle. Printed in U.S.A.
70-xxxx-xxxx-x

3M, ESPE and Filtek are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. © 3M 2014. All rights reserved. EsthetX, Grandio, Premise, Revolution Formula 2, SureFil SDR, Tetric EvoFlow, TPH and Venus are not trademarks of 3M.

Filtek™ Bulk Fill

Flowable Restorative

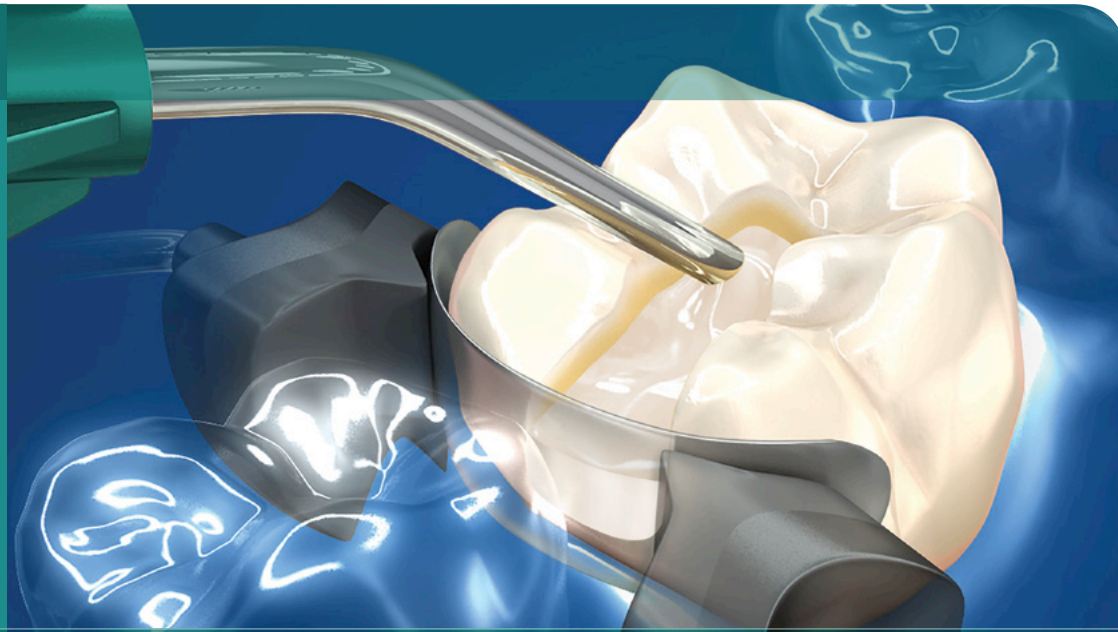

The bulk fill flowable
that's fast and strong.

Filtek™ Bulk Fill

Flowable Restorative

Reduce procedure time and increase your confidence with this bulk fill flowable.



With the trend toward faster and easier procedures, you have more choices than ever for a flowable composite. So how do you choose the one that stands up to your needs and expectations for efficiency, strength and all-around reliable performance?


Rely on the reputation of a leading innovator of dental restorative solutions ... 3M ESPE.

Excellent ✓
handling

Excellent as a liner

Flowable composites are designed to adapt easily, reducing the chance for voids and providing a good marginal seal to reduce the risk of post-operative sensitivity. These benefits are likely why 75% of dentists surveyed use a flowable composite as a liner.¹

Very
Low ↓
polymerization stress
compared to traditional flowables
placed and cured in 2mm increments²

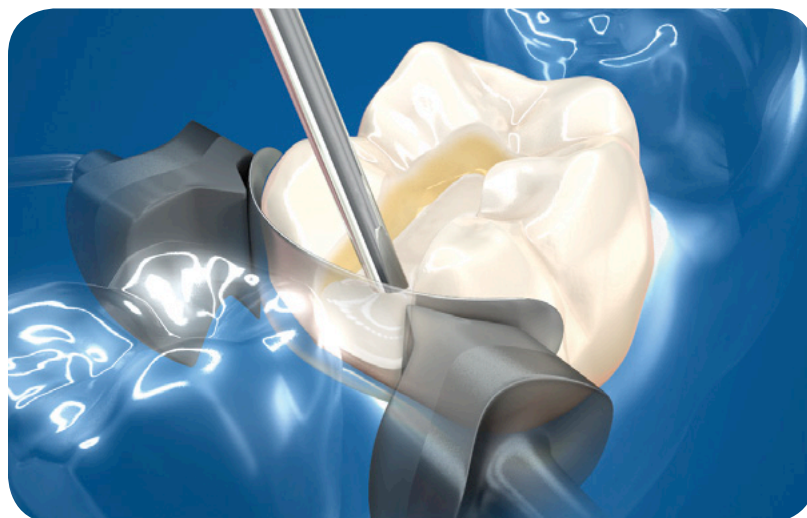

Easy
cavity
adaptation





1. Clinicians Report, July 2013 Vol. 6, Issue 7.

2. Compared to Grandio Flow, TPH3 Flow, Premise Flowable, Revolution Formula 2, EsthetX Flow and Tetric EvoFlow. Source: 3M ESPE internal data.


Excellent as a time-saving bulk fill base in deep restorations

Filtek Bulk Fill Flowable Restorative is formulated for low stress and an increased depth of cure, so it can be placed with confidence in one convenient 4 mm increment. The restoration is then completed with a minimum 2 mm thickness of a posterior or universal composite such as Filtek™ Supreme Ultra Universal Restorative.

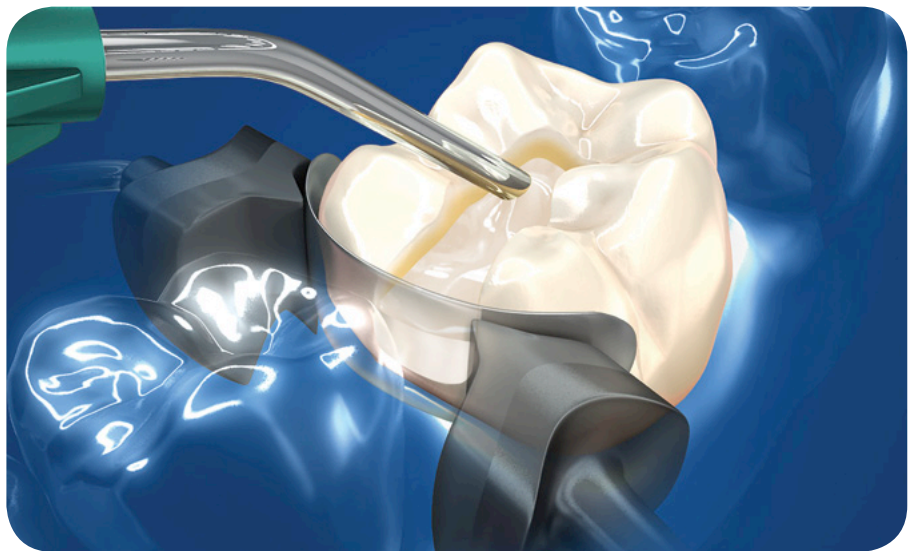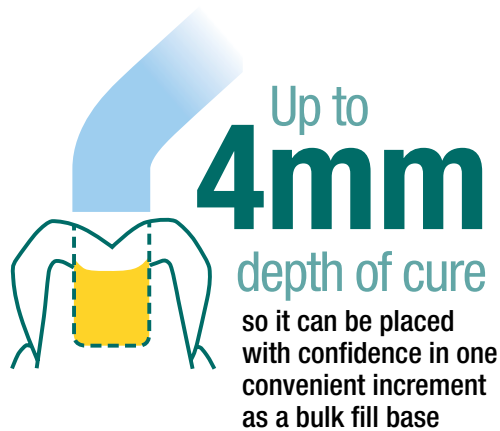

Less shrinkage stress

than more highly filled, sculptable composites⁴

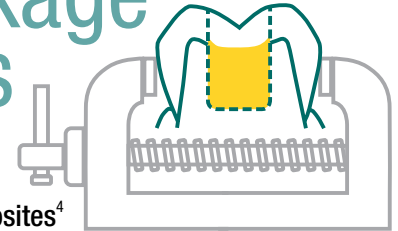

Highly Radiopaque

compared to traditional flowables and other bulk fill flowables³



Excellent for surface restorations

High strength and excellent wear resistance make it a versatile solution for surface restorations including small, non-stress bearing occlusal restorations.

Over
3X the wear
resistance
of the leading bulk fill flowable⁵

3. 3M ESPE internal data

4. Source: J Dent Res 92 (Spec Issue A):2439 El-Damanhoury.

5. In-vitro data on file. Source: J Dent Res 92 (Spec Iss A):abs. #2441 K. SAWLANI, P. BECK, L.C. RAMP, D. CAKIR-USTUN, and J. BURGESS, University of Alabama at Birmingham, Birmingham, AL.

6. 3M ESPE internal data. Compared to Surefill SDR Flow.